Addendum

Multilingual edition

for

cobas b 101 system
cobas b 101 system PST

Operator's Manual

Version 1.0

Addendum

*07758332190 (01)
as a replacement for*

cobas b 101 system

06630979018 (01) EN

cobas b 101 system PST

07042108018 (01) EN

Operator's Manual

Version 1.0

Document information

Revision history

Manual version	Revision date	Change description
07758332190 (01)	September 2015	Barcode scanner added.

Revision history

Publication changes

11.3.3 Accessories

Item	Comment				
External printer	- TLP2874 (Zebra) - CT-S281L (Citizen Systems) - Printer paper and labels				
Barcode scanner	- MS180-1UVG (Unitech corporation) - TD1120-BK-90 (Datalogic corporation) with USB cable 90A052044 The supported barcode scanners are preconfigured for usage of cobas b 101 system or cobas b 101 system PST. The preconfiguration of the barcode scanners must not be changed. The barcode scanners are able to read the following barcode formats: - Code 128 (ISO/IEC15417) - Code 93 (USS-CODE93) - Code 39 (ISO/IEC16388) - Interleaved 2 of 5 (ISO/IEC16390) - codabar [NW-7] (ANSI/AIM BC3-1995) Make sure you use one of the above barcode formats for your barcodes.  If the barcode for patient ID and/or operator ID is based on codabar [NW-7], Interleaved 2 of 5, or Code39, do not add the check digit to the barcode.  CAUTION Barcode scanner malfunction due to altering their preconfiguration Changing the barcode scanner preconfiguration may lead to their malfunction. - Do not change the preconfiguration of the barcode scanner. - For detailed information on usage, caution for barcode scanner itself, and trouble shooting, refer to the barcode scanner's own documentation.  CAUTION Barcode scanner malfunction due to using unsuitable cable Using an unsuitable barcode scanner cable may lead to malfunction. - Only use the cable that is supplied with the barcode scanner. <tr> <td>Base Unit Hub</td><td>Required for stand-alone network connection.</td></tr> <tr> <td>Optical Check Disc</td><td>Replacement disc if damaged or lost.</td></tr>	Base Unit Hub	Required for stand-alone network connection.	Optical Check Disc	Replacement disc if damaged or lost.
Base Unit Hub	Required for stand-alone network connection.				
Optical Check Disc	Replacement disc if damaged or lost.				

Table 23 Accessories

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