

Elecsys Vitamin B12 II

REF		SYSTEM
07212771 160	100	MODULAR ANALYTICS E170 cobas e 411 cobas e 601 cobas e 602

English

For use in the USA only

System information

For **cobas e 411** analyzer: test number 1410
For MODULAR ANALYTICS E170, **cobas e 601** and **cobas e 602** analyzers: Application Code Number 224

Intended use

Binding assay for the in vitro quantitative determination of vitamin B₁₂ in human serum and plasma. Measurements obtained by this device are used in the diagnosis and treatment of anemias of gastrointestinal malabsorption.

The electrochemiluminescence immunoassay "ECLIA" is intended for use on Elecsys and **cobas e** immunoassay analyzers.

Summary

Vitamin B₁₂, also referred to as cobalamin, is a complex organometallic compound in which a cobalt atom is situated within a corrin ring. It is a water-soluble vitamin which is synthesized by microorganisms. It cannot be synthesized in the human body and is seldom found in products of plant origin. Main sources of vitamin B₁₂ are meat, fish, eggs and dairy products.¹ The uptake in the gastrointestinal tract depends on intrinsic factor, which is synthesized by the gastric parietal cells, and on the "cobam receptor" in the distal ileum. The most frequent cause of severe vitamin B₁₂ deficiency is a lack of intrinsic factor due to autoimmune atrophic gastritis. The disease is historically called "pernicious anemia", even though many patients present with mainly neurologic manifestations. Examples of other causes for vitamin B₁₂ deficiency are malabsorption due to gastrectomy, inflammatory bowel disease or dietary deficiency, e.g. in strict vegetarians (vegans).²

Vitamin B₁₂ is the cofactor for two enzymes, methionine synthase and methylmalonyl CoA mutase.^{2,3} Methionine synthase, located in the cytoplasm, requires vitamin B₁₂ in the form of methylcobalamin and catalyzes the conversion of homocysteine to methionine, an essential amino acid. During this step a methyl group is transferred from methyltetrahydrofolate to the amino acid.³ This enzyme links the methylation pathway through synthesis of the methyl donor S-Adenosyl methionine and the pathway in which purine and pyrimidine are synthesized via generation of tetrahydrofolate.³ In the form of 5'-deoxyadenosylcobalamin, vitamin B₁₂ is also required for the mitochondrial enzyme methylmalonyl CoA mutase, which converts methylmalonyl CoA to succinyl CoA. This is a step in the oxidation of odd-chain fatty acids and catabolism of ketogenic amino acids.³ Thus, vitamin B₁₂ is important for DNA synthesis, regenerating methionine for protein synthesis and methylation, as well as for the development and initial myelination of the central nervous system (CNS) and for the maintenance of normal CNS function.^{2,3}

Vitamin B₁₂ deficiencies are common in wealthier countries principally among the elderly and are most prevalent in poorer populations. In general the prevalence increases with age.^{4,5}

Vitamin B₁₂ deficiency impacts red blood cell synthesis, resulting in megaloblastic anemia due to abnormal DNA synthesis.³ In addition it impairs neurological function, in particular demyelination of nerves in part due to abnormal methylation, leading to peripheral neuropathy, dementia, poor cognitive performance, and depression.³ Other effects of vitamin B₁₂ deficiency or depletion are increased risk of neural tube defects, osteoporosis, cerebrovascular and cardiovascular diseases.³ Early diagnosis is essential, because of the latent nature of this disorder and the risk of permanent neurological damage.^{3,5}

Generally, the primary test performed to confirm the diagnosis of vitamin B₁₂ deficiency is measurement of serum vitamin B₁₂ level.² Recent publications suggest that in addition the following biomarkers should be measured to improve the specificity of diagnosis: folate, methylmalonic acid (MMA), homocysteine and holotranscobalamin.^{2,5,6,7}

The Elecsys Vitamin B12 II assay employs a competitive test principle using intrinsic factor specific for vitamin B₁₂. Vitamin B₁₂ in the sample competes

with the added vitamin B₁₂ labeled with biotin for the binding sites on the ruthenium-labeled intrinsic factor complex^{a)}.

a) Tris(2,2'-bipyridyl)ruthenium(II)-complex (Ru(bpy)₃²⁺)

Test principle

Competition principle. Total duration of assay: 27 minutes.

- 1st incubation: By incubating the sample (15 µL) with the vitamin B₁₂ pretreatment 1 and pretreatment 2, bound vitamin B₁₂ is released.
- 2nd incubation: By incubating the pretreated sample with the ruthenium labeled intrinsic factor, a vitamin B₁₂-binding protein complex is formed, the amount of which is dependent upon the analyte concentration in the sample.
- 3rd incubation: After addition of streptavidin-coated microparticles and vitamin B₁₂ labeled with biotin, the still-vacant sites of the ruthenium labeled intrinsic factor become occupied, with formation of a ruthenium labeled intrinsic factor vitamin B₁₂ biotin complex. The entire complex becomes bound to the solid phase via interaction of biotin and streptavidin.
- The reaction mixture is aspirated into the measuring cell where the microparticles are magnetically captured onto the surface of the electrode. Unbound substances are then removed with ProCell/ProCell M. Application of a voltage to the electrode then induces chemiluminescent emission which is measured by a photomultiplier.
- Results are determined via a calibration curve which is instrument-specifically generated by 2-point calibration and a master curve provided via the reagent barcode or e-barcode.

Reagents - working solutions

The reagent rackpack (M, R1, R2) and the pretreatment reagents (PT1, PT2) are labeled as B12 II.

PT1 Pretreatment reagent 1 (white cap), 1 bottle, 4 mL:
Dithiothreitol 1.028 g/L; stabilizer, pH 5.5.

PT2 Pretreatment reagent 2 (gray cap), 1 bottle, 4 mL:
Sodium hydroxide 40 g/L; sodium cyanide 2.205 g/L.

M Streptavidin-coated microparticles (transparent cap), 1 bottle, 6.5 mL:
Streptavidin-coated microparticles 0.72 mg/mL; preservative.

R1 Intrinsic factor-Ru(bpy)₃²⁺ (gray cap), 1 bottle, 10 mL:
Ruthenium labeled recombinant porcine intrinsic factor 4 µg/L;
cobinamide dicyanide 15 µg/L; stabilizer; human serum albumin;
phosphate buffer, pH 5.5; preservative.

R2 Vitamin B₁₂-biotin (black cap), 1 bottle, 8.5 mL:
Biotinylated vitamin B12 25 µg/L; biotin 3 µg/L; phosphate buffer,
pH 7.0; preservative.

Precautions and warnings

For in vitro diagnostic use.

Exercise the normal precautions required for handling all laboratory reagents.

Disposal of all waste material should be in accordance with local guidelines. Safety data sheet available for professional user on request.

For USA: Caution: Federal law restricts this device to sale by or on the order of a physician.

This kit contains components classified as follows in accordance with the Regulation (EC) No. 1272/2008:



Danger

Elecsys Vitamin B12 II



- H290 May be corrosive to metals.
- H314 Causes severe skin burns and eye damage.
- H412 Harmful to aquatic life with long lasting effects.

Prevention:

- P273 Avoid release to the environment.
- P280 Wear protective gloves/ protective clothing/ eye protection/ face protection.

Response:

- P301 + P330 IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. + P331
- P303 + P361 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water. + P353
- P304 + P340 IF INHALED: Remove person to fresh air and keep comfortable for breathing. + P310 Immediately call a POISON CENTER/ doctor.
- P305 + P351 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. + P338 Continue rinsing. Immediately call a POISON CENTER/ doctor. + P310

Product safety labeling follows EU GHS guidance.
 Contact phone: 1-800-428-2336

All human material should be considered potentially infectious. All products derived from human blood are prepared exclusively from the blood of donors tested individually and shown to be free from HBsAg and antibodies to HCV and HIV. The testing methods used assays approved by the FDA or cleared in compliance with the European Directive 98/79/EC, Annex II, List A.

However, as no testing method can rule out the potential risk of infection with absolute certainty, the material should be handled with the same level of care as a patient specimen. In the event of exposure, the directives of the responsible health authorities should be followed.^{8,9}

Avoid foam formation in all reagents and sample types (specimens, calibrators and controls).

Reagent handling

The reagents in the kit have been assembled into a ready-for-use unit that cannot be separated.
 All information required for correct operation is read in from the respective reagent barcodes.

Storage and stability

Store at 2-8 °C.
 Do not freeze.
 Store the Elecsys reagent kit **upright** in order to ensure complete availability of the microparticles during automatic mixing prior to use.

Stability:	
unopened at 2-8 °C	up to the stated expiration date
after opening at 2-8 °C	12 weeks
on the analyzers	5 weeks onboard or 60 days when stored alternatively in the refrigerator and on the analyzer, with the total time onboard on the analyzer not exceeding 10 x 8 hours

Specimen collection and preparation

Only the specimens listed below were tested and found acceptable.

Serum collected using standard sampling tubes or tubes containing separating gel.
 Na-heparin, Li-heparin, K₂-EDTA and K₃-EDTA plasma. Li-heparin plasma tubes containing separating gel can be used.
 Criterion: Slope 0.9-1.1 + intercept within $\pm 2 \times$ Limit of Blank + coefficient of correlation ≥ 0.95 .
 Stable for 2 hours at 20-25 °C, 48 hours at 2-8 °C, 8 weeks at -20 °C (± 5 °C). Freeze once only.
 Stability of serum obtained with separating tubes: 24 hours at 2-8 °C (note the data provided by the tube manufacturer).

The sample types listed were tested with a selection of sample collection tubes or systems that were commercially available at the time of testing, i.e. not all available tubes of all manufacturers were tested. Sample collection systems from various manufacturers may contain differing materials which could affect the test results in some cases. When processing samples in primary tubes (sample collection systems), follow the instructions of the tube manufacturer.

Centrifuge samples containing precipitates before performing the assay.
 Do not use any hemolyzed samples because samples showing visible signs of hemolysis will have falsely low results.
 Do not use heat-inactivated samples.
 Do not use samples and controls stabilized with azide.

Vitamin B₁₂ determinations should be performed on serum or plasma samples from fasting patients.
 Ensure the samples, calibrators and controls are at 20-25 °C prior to measurement.

Due to possible evaporation effects, samples, calibrators and controls on the analyzers should be analyzed/measured within 2 hours.

The claims, including those pertaining to sample stability made in the labeling of the cleared/approved reagents of Roche Diagnostics are part of the clearance of the overall IVD test system (assay). Sample stability was tested only for the temperatures/time frame as claimed by the manufacturer under the conditions claimed in the method sheet. It is the responsibility of the individual laboratory to use all available references and/or its own studies to determine specific stability criteria for its laboratory.

Materials provided

See "Reagents – working solutions" section for reagents.

Materials required (but not provided)

- [REF] 07212780190, Vitamin B12 II CalSet, for 4 x 1 mL
 - [REF] 05618860160, PreciControl Varia, for 6 x 3 mL
 - [REF] 03183971122, Diluent Universal, 2 x 36 mL sample diluent
 - General laboratory equipment
 - MODULAR ANALYTICS E170 or **cobas e** analyzer
- Accessories for **cobas e** 411 analyzers:
- [REF] 11662988122, ProCell, 6 x 380 mL system buffer
 - [REF] 11662970122, CleanCell, 6 x 380 mL measuring cell cleaning solution
 - [REF] 11930346122, Elecsys SysWash, 1 x 500 mL washwater additive
 - [REF] 11933159001, Adapter for SysClean
 - [REF] 11706802001, AssayCup, 60 x 60 reaction cups
 - [REF] 11706799001, AssayTip, 30 x 120 pipette tips
 - [REF] 11800507001, Clean-Liner
- Accessories for MODULAR ANALYTICS E170, **cobas e** 601 and **cobas e** 602 analyzers:
- [REF] 04880340190, ProCell M, 2 x 2 L system buffer
 - [REF] 04880293190, CleanCell M, 2 x 2 L measuring cell cleaning solution
 - [REF] 03023141001, PC/CC-Cups, 12 cups to prewarm ProCell M and CleanCell M before use
 - [REF] 03005712190, ProbeWash M, 12 x 70 mL cleaning solution for run finalization and rinsing during reagent change
 - [REF] 03004899190, PreClean M, 5 x 600 mL detection cleaning solution

Elecsys Vitamin B12 II



- [REF] 12102137001, AssayTip/AssayCup, 48 magazines x 84 reaction cups or pipette tips, waste bags
- [REF] 03023150001, WasteLiner, waste bags
- [REF] 03027651001, SysClean Adapter M

Accessories for all analyzers:

- [REF] 11298500160, ISE Cleaning Solution/Elecsys SysClean, 5 x 100 mL system cleaning solution

Assay

For optimum performance of the assay follow the directions given in this document for the analyzer concerned. Refer to the appropriate operator's manual for analyzer-specific assay instructions.

Resuspension of the microparticles takes place automatically prior to use. Read in the test-specific parameters via the reagent barcode. If in exceptional cases the barcode cannot be read, enter the 15-digit sequence of numbers (except for the **cobas e 602** analyzer).

MODULAR ANALYTICS E170, **cobas e 601** and **cobas e 602** analyzers: PreClean M solution is necessary.

Bring the cooled reagents to approximately 20 °C and place on the reagent disk (20 °C) of the analyzer. Avoid foam formation. The system automatically regulates the temperature of the reagents and the opening/closing of the bottles.

Calibration

Traceability: This method has been standardized against the Elecsys Vitamin B12 assay ([REF] 04745736).

Accuracy to WHO Standard 03/178:

A study was performed to evaluate the accuracy of the Elecsys Vitamin B12 II assay using the Vitamin B12 World Health Organization International Standard 03/178.¹⁰ Two reagent lots were used on 16 instruments. The mean recovery of the target value of WHO IS 03/178 (480 pg/mL) was 102 %.

Every Elecsys reagent set has a barcoded label containing specific information for calibration of the particular reagent lot. The predefined master curve is adapted to the analyzer using the relevant CalSet.

Calibration frequency: Calibration must be performed once per reagent lot using fresh reagent (i.e. not more than 24 hours since the reagent kit was registered on the analyzer).

Calibration interval may be extended based on acceptable verification of calibration by the laboratory.

Renewed calibration is recommended as follows:

- after 1 month (28 days) when using the same reagent lot
- after 7 days (when using the same reagent kit on the analyzer)
- as required: e.g. quality control findings outside the defined limits

Quality control

For quality control, use PreciControl Varia.

In addition, other suitable control material can be used.

Controls for the various concentration ranges should be run individually at least once every 24 hours when the test is in use, once per reagent kit, and following each calibration.

The control intervals and limits should be adapted to each laboratory's individual requirements. Values obtained should fall within the defined limits. Each laboratory should establish corrective measures to be taken if values fall outside the defined limits.

If necessary, repeat the measurement of the samples concerned.

Follow the applicable government regulations and local guidelines for quality control.

Calculation

The analyzer automatically calculates the analyte concentration of each sample (either in pmol/L or pg/mL).

Conversion factors: pmol/L x 1.36 = pg/mL

pg/mL x 0.738 = pmol/L

Limitations - interference

The assay is unaffected by icterus (bilirubin $\leq 1112 \mu\text{mol/L}$ or $\leq 65 \text{ mg/dL}$), hemolysis (Hb $\leq 0.025 \text{ mmol/L}$ or $\leq 0.04 \text{ g/dL}$; samples showing visible signs of hemolysis may cause interference), lipemia (Intralipid

$\leq 17.1 \text{ mmol/L}$ or $\leq 1500 \text{ mg/dL}$), biotin ($\leq 205 \text{ nmol/L}$ or $\leq 50 \text{ ng/mL}$), IgG $\leq 28 \text{ g/L}$, IgA $\leq 16 \text{ g/L}$ and IgM $\leq 10 \text{ g/L}$.

Falsely low results are obtained when using samples with hemoglobin concentrations $> 0.04 \text{ g/dL}$.

Criterion: Recovery within $\pm 10 \%$ of initial value with samples $> 200 \text{ pg/mL}$ and $\leq \pm 20 \text{ pg/mL}$ with samples $\leq 200 \text{ pg/mL}$.

Samples should not be taken from patients receiving therapy with high biotin doses (i.e. $> 5 \text{ mg/day}$) until at least 8 hours following the last biotin administration.

No interference was observed from rheumatoid factors up to a concentration of 1500 IU/mL .

In vitro tests were performed on 16 commonly used pharmaceuticals. No interference with the assay was found.

In rare cases, interference due to extremely high titers of antibodies to analyte-specific antibodies, streptavidin or ruthenium can occur. These effects are minimized by suitable test design.

Because intrinsic factor is typically used as the binding protein in serum vitamin B₁₂ assays, anti-intrinsic factor antibodies (which are common in pernicious anemia) can lead to elevated vitamin B₁₂ measurement values.^{2,11,12} The Elecsys Vitamin B12 II assay is designed to avoid interference due to anti-intrinsic factor antibodies.¹³

Samples with extremely high total protein concentrations (hyperproteinemia) are not suitable for use in this assay. Hyperproteinemia may be caused by, but not limited to, the following conditions: Lymphoma,^{14,15} bone marrow disorders such as multiple myeloma, monoclonal gammopathy of undetermined significance (MGUS), Waldenström macroglobulinemia, plasmocytoma,^{14,15,16,17,18,19,20} Amyloidosis.^{20,21} Respective samples may lead to the formation of protein gel in the assay cup, which may cause a run abort. The critical total protein concentration is dependent upon the individual sample composition.

For diagnostic purposes, the results should always be assessed in conjunction with the patient's medical history, clinical examination and other findings.

Note: The presence of immunoglobulin-vitamin B₁₂ complexes may cause unexpectedly high values of vitamin B₁₂.^{22,23}

Limits and ranges

Measuring range

150-2000 pg/mL or 111-1476 pmol/L (defined by the Limit of Quantitation and the maximum of the master curve). Values below the Limit of Quantitation are reported as $< 150 \text{ pg/mL}$ or $< 111 \text{ pmol/L}$. Values above the measuring range are reported as $> 2000 \text{ pg/mL}$ or $> 1476 \text{ pmol/L}$.

Lower limits of measurement

Limit of Blank, Limit of Detection and Limit of Quantitation

Limit of Blank = 50 pg/mL (36.9 pmol/L)

Limit of Detection = 100 pg/mL (73.8 pmol/L)

Limit of Quantitation = 150 pg/mL (111 pmol/L) with an allowable imprecision of $\leq 20 \%$

The Limit of Blank, Limit of Detection and Limit of Quantitation were determined in accordance with the CLSI (Clinical and Laboratory Standards Institute) EP17-A requirements.

The Limit of Blank is the 95th percentile value from $n \geq 60$ measurements of analyte-free samples over several independent series. The Limit of Blank corresponds to the concentration below which analyte-free samples are found with a probability of 95 %.

The Limit of Detection is determined based on the Limit of Blank and the standard deviation of low concentration samples. The Limit of Detection corresponds to the lowest analyte concentration which can be detected (value above the Limit of Blank with a probability of 95 %).

The Limit of Quantitation is defined as the lowest amount of analyte in a sample that can be accurately quantitated with a total allowable relative error of $\leq 20 \%$.

It has been determined using low concentration vitamin B₁₂ samples.

Dilution

Samples with vitamin B₁₂ concentrations above the measuring range can be manually diluted 1:2 with Diluent Universal. The concentration of the diluted sample must be $> 738 \text{ pmol/L}$ or $> 1000 \text{ pg/mL}$.

After manual dilution, multiply the result by the dilution factor.

Elecsys Vitamin B12 II



Note: Sample-dependent non-linearity upon dilution is seen with samples having analyte levels beyond the measuring range. As Diluent Universal may contain low levels of endogenous vitamin B₁₂, it is recommended that linearity studies be performed using a known low analyte-containing serum pool. Samples outside the measuring range can be diluted 1:2 with Diluent Universal; the effect of endogenous vitamin B₁₂ concentration is insignificant at these levels.

Expected values

Because differences may exist with respect to population and dietary status, it is recommended that normal ranges be determined by each laboratory over a suitable period of time and in a statistically significant number of assays before clinical significance is attached to the results of these tests.

The values shown in the following table were performed on samples from an apparently healthy population, using the Elecsys Vitamin B12 II assay. The calculation is based on 120 sera (66 men, 54 women). The age range was between 22 and 79 years. Pregnant women were excluded. The reference population was selected according to normal homocysteine values.

N	Median		Range (2.5 th -97.5 th percentile)	
	pg/mL	pmol/L	pg/mL	pmol/L
120	443	327	232-1245	171-919

These values should only be used as guidelines.

Each laboratory should investigate the transferability of the expected values to its own patient population and if necessary determine its own reference ranges.

Specific performance data

Representative performance data on the analyzers are given below. Results obtained in individual laboratories may differ.

Precision

Precision was determined using Elecsys reagents, pooled human sera and controls in a protocol (EP5-A2) of the CLSI (Clinical and Laboratory Standards Institute): 2 runs per day in duplicate each for 21 days (n = 84). The following results were obtained:

cobas e 411 analyzer					
Sample	Mean	Repeatability		Intermediate precision	
		SD	CV	SD	CV
	pg/mL	pg/mL	%	pg/mL	%
Human serum 1	176	8.86	5.0	12.7	7.2
Human serum 2	405	13.0	3.2	17.5	4.3
Human serum 3	960	19.7	2.1	31.0	3.2
Human serum 4	1230	27.4	2.2	46.4	3.8
Human serum 5	1940	40.9	2.1	72.6	3.7
PreciControl Varia 0	229	8.96	3.9	12.4	5.4
PreciControl Varia 1	447	12.2	2.7	18.6	4.2
PreciControl Varia 2	934	20.2	2.2	38.4	4.1

cobas e 411 analyzer					
Sample	Mean	Repeatability		Intermediate precision	
		SD	CV	SD	CV
	pmol/L	pmol/L	%	pmol/L	%
Human serum 1	130	6.54	5.0	9.37	7.2
Human serum 2	299	9.59	3.2	12.9	4.3
Human serum 3	708	14.5	2.1	22.9	3.2
Human serum 4	908	20.2	2.2	34.2	3.8
Human serum 5	1432	30.2	2.1	53.6	3.7

cobas e 411 analyzer					
Sample	Mean	Repeatability		Intermediate precision	
		SD	CV	SD	CV
	pmol/L	pmol/L	%	pmol/L	%
PreciControl Varia 0	169	6.61	3.9	9.15	5.4
PreciControl Varia 1	330	9.00	2.7	13.7	4.2
PreciControl Varia 2	689	14.9	2.2	28.3	4.1

MODULAR ANALYTICS E170, cobas e 601 and cobas e 602 analyzers					
Sample	Mean	Repeatability		Intermediate precision	
		SD	CV	SD	CV
	pg/mL	pg/mL	%	pg/mL	%
Human serum 1	176	5.84	3.3	9.14	5.2
Human serum 2	407	8.24	2.0	12.7	3.1
Human serum 3	1010	13.2	1.3	21.1	2.1
Human serum 4	1230	19.8	1.6	28.8	2.3
Human serum 5	1890	29.8	1.6	41.5	2.2
PreciControl Varia 0	196	4.47	2.3	10.1	5.1
PreciControl Varia 1	448	7.16	1.6	15.3	3.4
PreciControl Varia 2	917	12.0	1.3	27.8	3.0

MODULAR ANALYTICS E170, cobas e 601 and cobas e 602 analyzers					
Sample	Mean	Repeatability		Intermediate precision	
		SD	CV	SD	CV
	pmol/L	pmol/L	%	pmol/L	%
Human serum 1	130	4.31	3.3	6.75	5.2
Human serum 2	300	6.08	2.0	9.37	3.1
Human serum 3	745	9.74	1.3	15.6	2.1
Human serum 4	908	14.6	1.6	21.3	2.3
Human serum 5	1395	22.0	1.6	30.3	2.2
PreciControl Varia 0	145	3.30	2.3	7.45	5.1
PreciControl Varia 1	331	5.28	1.6	11.3	3.4
PreciControl Varia 2	677	8.86	1.3	20.5	3.0

Method comparison

A comparison of the Elecsys Vitamin B12 assay (calibrated with Vitamin B12 CalSet II; x) and the Elecsys Vitamin B12 II assay (calibrated with Vitamin B12 II CalSet; y) using clinical samples gave the following correlations (pg/mL):

Number of samples measured: 120

Passing/Bablok²⁴ Linear regression

$$y = 0.950x - 9.83$$

$$y = 0.984x - 26.7$$

$$\tau = 0.941$$

$$r = 0.996$$

The sample concentrations were between 156 and 1753 pg/mL (115 and 1294 pmol/L).

Linearity

Ten dilutions of a high analyte serum sample were tested in triplicate to demonstrate the linear range of 150-2000 pg/mL. The linearity data were analyzed with regards to linear, quadratic and cubic polynomials according to CLSI EP6-A. The mean value for each dilution was used for polynomial regression analysis.

$$y = 1.071x - 14.552$$

$$r^2 = 0.998$$

Elecsys Vitamin B12 II



Analytical specificity

The following cross-reactivities were found, tested with vitamin B₁₂ concentrations of 129 pg/mL and 550 pg/mL.

Cross-reactant	Maximum concentration tested ng/mL	Cross-reactivity %
Cobinamide dicyanide	210	0.003







References

- Thomas L. Clinical Laboratory Diagnostics: Use and Assessment of Clinical Laboratory Results; 1st Edition, Frankfurt/Main: TH-Books-Verl.-Ges., 1998:424-431.
- Stabler SP. Vitamin B12 deficiency. N Engl J Med 2013;368:149-160.
- Allen LH. Vitamin B-12. Adv Nutr 2012;3(1):54-55. doi: 10.3945/an.111.001370. Epub 2012 Jan 5.
- Allen LH. How common is vitamin B-12 deficiency? Am J Clin Nutr 2009;89(2):693S-696S. Epub 2008 Dec 30.
- Chatthanawaree W. Biomarkers of cobalamin (vitamin B12) deficiency and its application. J Nutr Health Aging 2011 Mar;15(3):227-313.
- Yetley EA, Pfeiffer CM, Phinney KW, et al., Biomarkers of vitamin B-12 status in NHANES: a roundtable summary. Am J Clin Nutr 2011 Jul;94(1):313S-321S.
- Hvas AM, Nexø E. Diagnosis and treatment of vitamin B12 deficiency - an update. Haematologica 2006;91(11):1506-1512.
- Occupational Safety and Health Standards: Bloodborne pathogens. (29 CFR Part 1910.1030). Fed. Register.
- Directive 2000/54/EC of the European Parliament and Council of 18 September 2000 on the protection of workers from risks related to exposure to biological agents at work.
- Thorpe SJ, Heath A, Blackmore S, et al. International Standard for serum vitamin B12 and serum folate: international collaborative study to evaluate a batch of lyophilised serum for B12 and folate content. Clin Chem Lab Med 2007;45(3):380-386.
- Yang DT, Cook RJ. Spurious elevations of vitamin B12 with pernicious anemia. N Engl J Med 2012;366:1742-1743.
- Carmel R, Agrawal YP. Failures of cobalamin assays in pernicious anemia. N Engl J Med 2012;367:385-386. [Erratum, N Engl J Med 2012;367:976.]
- Schilling KA, Wiesgigl M. The Elecsys® Vitamin B12 assay is not affected by anti-intrinsic factor antibodies. Clin Chem Lab Med 2013 Jun 29;51(11):e251-e252.
- Wu AHB. Tietz clinical guide to laboratory tests, 4th ed. St. Louis, Saunders/Elsevier 2006:608-609, 916-917.
- Paricaud K, Moulis G, Combis MS, et al. Causes of prothidemia above 100 g/L. Eur J Intern Med 2014;25:e123.
- Filippatos TD, Liamis G, Christopoulou F, et al. Ten common pitfalls in the evaluation of patients with hyponatremia. Eur J Intern Med 2016;29:22-25.
- Mailankody S, Landgren O. Monoclonal gammopathy of undetermined significance and Waldenström's macroglobulinemia. Best Pract Res Clin Haematol 2016;29:187-193.
- Morel P, Duhamel A, Gobbi P, et al. International prognostic scoring system for Waldenström macroglobulinemia. Blood 2009;113:4163-4170.
- Rajkumar SV. Multiple Myeloma. Curr Probl Cancer 2009;33:7-64.
- Gertz MA. Immunoglobulin light chain amyloidosis: 2016 update on diagnosis, prognosis, and treatment. Am J Hematol 2016;91:947-956.
- Wu AHB. Tietz clinical guide to laboratory tests, 4th ed. St. Louis, Saunders/Elsevier 2006: 916-917, 925.
- Jeffery J, Millar H, MacKenzie P, et al. An IgG complexed form of vitamin B12 is a common cause of elevated serum concentrations. Clin Biochem 2010 Jan;43(1-2):82-88. doi: 10.1016/j.clinbiochem.2009.08.022. Epub 2009 Sep 8.
- Bowen RA, Drake SK, Vanjani R, et al. Markedly increased vitamin B12 concentrations attributable to IgG-IgM-vitamin B12 immune complexes. Clin Chem 2006;52(11):2107-2114.
- Bablok W, Passing H, Bender R, et al. A general regression procedure for method transformation. Application of linear regression procedures for method comparison studies in clinical chemistry, Part III. J Clin Chem Clin Biochem 1988 Nov;26(11):783-790.

For further information, please refer to the appropriate operator's manual for the analyzer concerned, the respective application sheets, the product information and the Method Sheets of all necessary components (if available in your country).

Symbols

Roche Diagnostics uses the following symbols and signs in addition to those listed in the ISO 15223-1 standard (for USA: see <https://usdiagnostics.roche.com> for definition of symbols used):

	Contents of kit
	Analyzers/Instruments on which reagents can be used
	Reagent
	Calibrator
	Volume after reconstitution or mixing
	Global Trade Item Number

FOR US CUSTOMERS ONLY: LIMITED WARRANTY


Roche Diagnostics warrants that this product will meet the specifications stated in the labeling when used in accordance with such labeling and will be free from defects in material and workmanship until the expiration date printed on the label. THIS LIMITED WARRANTY IS IN LIEU OF ANY OTHER WARRANTY, EXPRESS OR IMPLIED, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR PARTICULAR PURPOSE. IN NO EVENT SHALL ROCHE DIAGNOSTICS BE LIABLE FOR INCIDENTAL, INDIRECT, SPECIAL OR CONSEQUENTIAL DAMAGES.

COBAS, COBAS E, ELECSYS and PRECICONTROL are trademarks of Roche. INTRALIPID is a trademark of Fresenius Kabi AB.

All other product names and trademarks are the property of their respective owners.

Additions, deletions or changes are indicated by a change bar in the margin.

© 2018, Roche Diagnostics

 Roche Diagnostics GmbH, Sandhofer Strasse 116, D-68305 Mannheim
www.roche.com

Distribution in USA by:
Roche Diagnostics, Indianapolis, IN
US Customer Technical Support 1-800-428-2336

