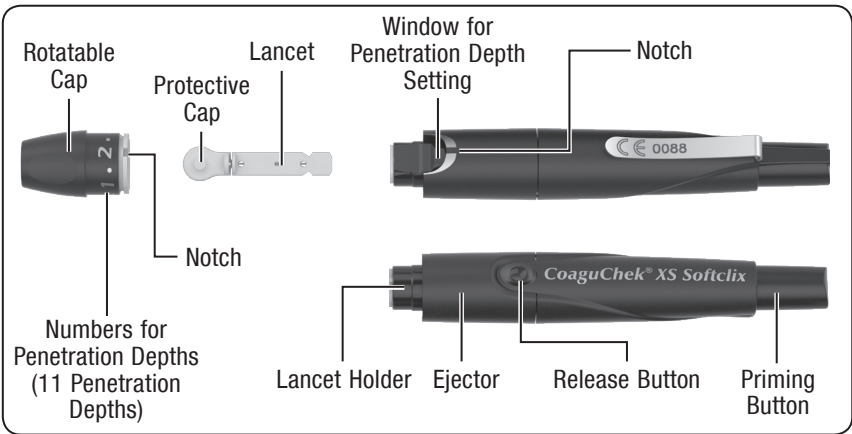
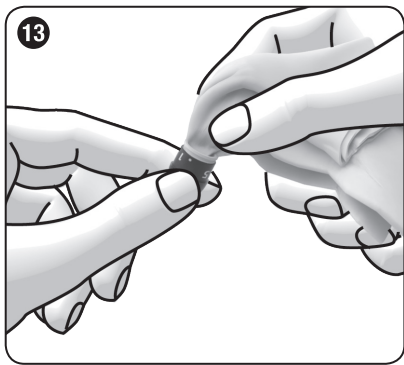
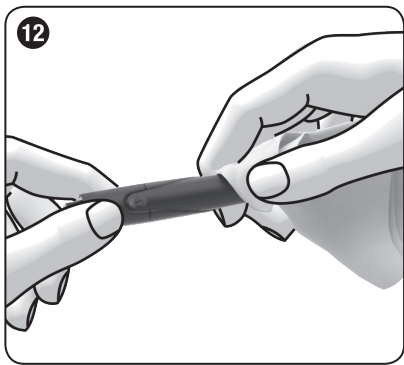
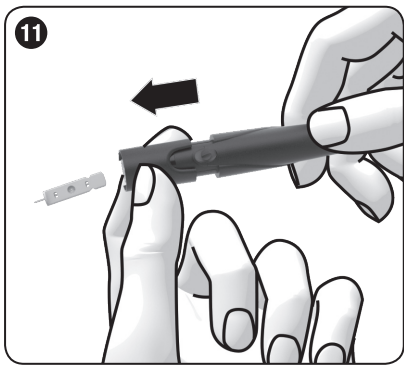
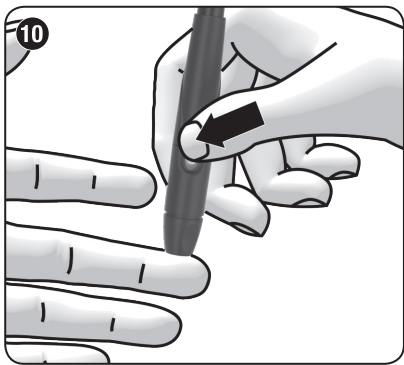
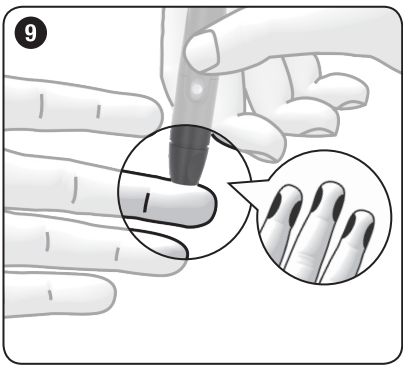
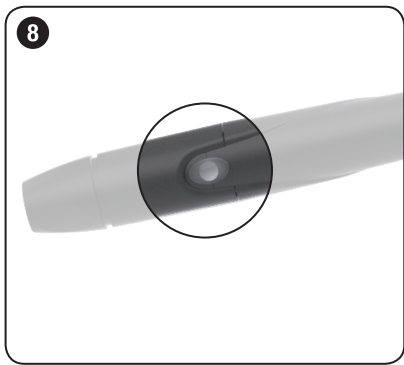
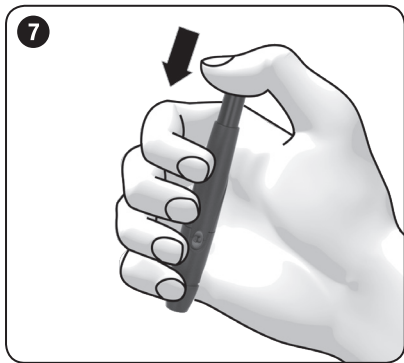
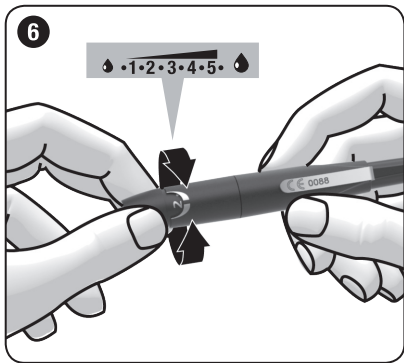
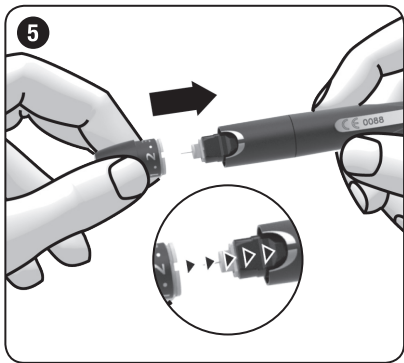
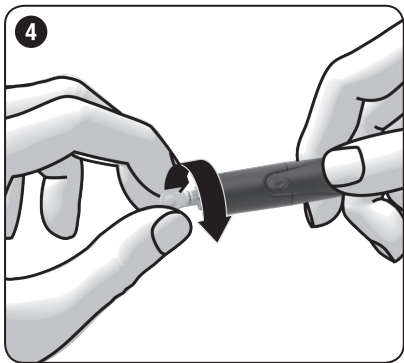
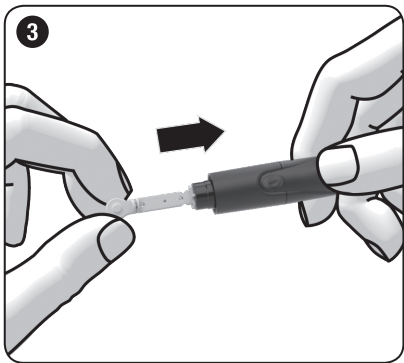
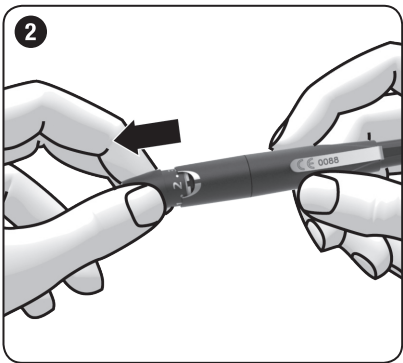
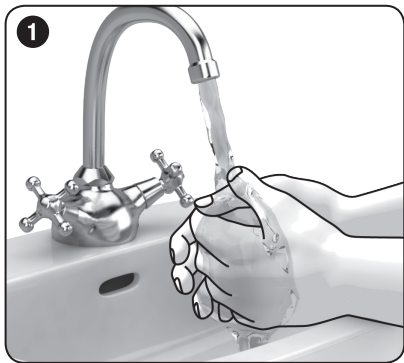
CoaguChek® XS Softclix

Lancing device

Instructions for use

Intended Use

The lancing device is intended for the hygienic collection of capillary blood for testing purposes from the side of a fingertip. The lancing device is for use only on a single patient.

For use only on a single patient. Disinfect reusable components according to manufacturer's instructions between each use.

Each lancet may only be used once to obtain blood. Used lancets must be safely discarded after a single use.

WARNING
Do not use on more than one patient. Improper use of blood lancets can increase the risk of inadvertent transmission of bloodborne pathogens, particularly in settings where multiple patients are tested. The cleaning and disinfection instructions for this device are intended only to reduce the risk of local use site infection; they cannot render this device safe for use for more than one patient.

WARNING

- If the lancing device is being operated by a second person who is providing testing assistance to the user, the lancing device should be cleaned and disinfected prior to use by the second person. The second person must wear protective gloves for all the steps when providing testing assistance.
- Keep the lancing device with all its components away from all children. There is a risk of suffocation if small parts (e.g. caps or similar objects) are swallowed.
- Only insert CoaguChek® Softclix or Accu-Chek® Softclix lancets in the CoaguChek® XS Softclix lancing device. Other lancets may impair the operation of the lancing device or damage it permanently.
- If the lancing device falls with a lancet inserted, the lancet may become loose in the lancet holder. You cannot use it anymore to obtain blood. In rare cases, the lancet could protrude from the opening in the cap and you could injure yourself on it. Therefore, do not touch the front of the cap. If the

lancing device falls, pick it up with care. Remove the cap from the lancing device. Pay attention to the lancet so that you do not injure yourself on it. Always eject the lancet and discard it.

Features

The CoaguChek XS Softclix lancing device is a device for virtually pain-free collection of capillary blood from the fingertips. The lancing device has 11 different penetration depths so that you can adjust the penetration depth to the texture of your skin. You insert a lancet into the lancing device. With the priming button, you prime the lancing device. With the release button, you trigger a lancet. When a lancet is triggered, it penetrates the skin. Afterwards, the lancet retracts automatically into the lancing device. This helps prevent accidental fingersticks.

Before Using the Lancing Device

- Wash your hands with warm water and soap (see 1).

Inserting the Lancet

You have to insert a lancet to be able to obtain blood with the lancing device.

- Remove the cap from the lancing device (see 2).
- Insert a new lancet into the lancet holder as far as it will go (see 3). The lancet must audibly click into place.
- Twist the protective cap off of the lancet (see 4).
- Place the cap back on again, lining up the notch on the cap with the notch on the lancing device (see 5). The cap must audibly click into place.

Setting the Penetration Depth

You can set 11 penetration depths (0.5 to 5.5). The penetration depth set is shown in the window. The larger the number, the greater the penetration depth.

If you have no experience with this lancing device, we recommend setting the penetration depth to 5.

- Rotate the cap until the desired penetration depth is set (see 6).

Priming the Lancing Device

In order to be able to obtain blood, you must prime the lancing device.

- Press priming button down as far as it will go (see 7). Do not press the release button simultaneously.

The lancing device is primed when the center of the release button is yellow (see 8).

WARNING
Only prime the lancing device just before obtaining blood.

Obtaining a Blood Drop

In general, blood can be obtained from any finger. Certain fingers can be unsuitable if, for example, a skin or fingernail infection is present. We recommend obtaining capillary blood from the sides of the fingertips as these areas are the least sensitive to pain.

WARNING

- Always use a new lancet when you obtain blood. In doing this, you reduce the risk of infection, and obtaining blood remains virtually pain free.
- Only use the lancing device when the cap is attached. Without the cap, the lancet penetrates too deeply and may cause discomfort.
- Wash your hands with warm water and soap before obtaining blood (see 1). Dry your hands well. This reduces contamination of the puncture site and stimulates blood flow.

- Disinfect the selected puncture site. The puncture site must dry completely afterwards.
- Press the lancing device firmly against the selected puncture site (see 9).
- Press the release button (see 10).

The lancet is released and it penetrates the skin.

- Massage the finger in the direction of the fingertip to encourage a blood drop to form.

WARNING

Do not press or squeeze the finger. This can cause tissue fluid to mix with the blood and can falsify the coagulation values.

The amount of blood that emerges depends on the penetration depth and the pressure used to hold the lancing device against the skin.

If not enough blood emerges, apply more pressure on the lancing device when you next obtain blood. If that is not sufficient, increase the penetration depth progressively in addition.

If too much blood emerges, decrease the penetration depth.

- Test your coagulation value immediately after you obtained blood, according to the User's Manual for your meter.

We recommend protecting the puncture site from contamination afterwards.

Ejecting the Used Lancet

- Remove the cap from the lancing device (see 2).

- Slide the ejector forward (see 11).

The used lancet is ejected.

- Place the cap back on again (see 5).
- Dispose of used lancets according to local regulations.

WARNING

Dispose of used lancets in such a way as to prevent injury from the needles.

After Using the Lancing Device

- Wash your hands with warm water and soap (see 1).

Storing and Using the Lancing Device

Protect the lancing device from excessive humidity, sunlight and heat. Store and use at room temperature.

WARNING

Store the lancing device only without a lancet inserted.

Cleaning and Disinfecting the Lancing Device and Cap

The lancing device can spread infection if it becomes soiled with blood.^{1,2} During normal testing any lancing device may come in contact with blood. For this reason, it is important to keep the lancing device clean and disinfected.

To clean and disinfect without damaging the lancing device, follow these procedures carefully.

What Is the Difference between Cleaning and Disinfecting?

- Cleaning is the removal of dirt from the lancing device.³
- Disinfecting is the removal of most, but not all, disease-causing and other types of microorganisms from the meter or lancing device.³

Approved Cleaning and Disinfecting Product

The following product has been approved for cleaning and disinfecting the lancing device:

Super Sani-Cloth (EPA* reg. no. 9480-4
* Environmental Protection Agency

Super Sani-Cloth can be purchased from Amazon.com, Officedepot.com, and Walmart.com.

- Do not use any other cleaning or disinfecting solutions. Using solutions other than the Super Sani-Cloth could result in damage to the lancing device.
- The effect of using more than one product interchangeably to clean and disinfect the lancing device has not been tested. Always use Super Sani-Cloth to clean and disinfect the lancing device.
- Roche has tested the approved product with 260 cycles each for cleaning and disinfecting, for a total of 520 cycles, which is equal to cleaning and disinfecting once per week for a 5-year period.

NOTE

For technical assistance or questions on cleaning and disinfecting, call CoaguChek Technical Service at 1-800-428-4674.

When to Clean and Disinfect the Lancing Device

- Clean the lancing device to remove visible dirt or other material prior to disinfecting.
- Clean and disinfect the lancing device at least once per week to remove visible dirt or other material for safe handling.
- Clean and disinfect the lancing device before allowing anyone else to handle the lancing device, for instance, if you have someone assisting you. Do not allow anyone else to use the lancing device to obtain their blood sample.

NOTE

- Do not throw away the cap after each use.
- Always remove the lancet before cleaning and disinfecting the lancing device.
- Using cleaning and disinfecting products could result in damage to the lancing device. If you notice any of the following signs of deterioration after cleaning and disinfecting your lancing device, stop using your lancing device and call CoaguChek Technical Service at 1-800-428-4674: residue around buttons, difficulty in priming the lancing device, difficulty in inserting the lancet.
- You might observe a slight discoloration of the lancing device after multiple cleaning and disinfecting cycles. This does not affect the functionality of the device.

What to Clean and Disinfect

The following parts of the lancing device should be cleaned and disinfected:

- The entire lancing device surface
- The cap

How to Clean and Disinfect the Lancing Device

WARNING

Failure to follow these instructions may damage the lancing device and stop it from working properly.

- Do not get any moisture into openings.
- Always use the same product for both cleaning and disinfecting. A separate Super Sani-Cloth should be used for cleaning and disinfecting.
- ① Wash hands thoroughly with soap and water.
- ② Wipe the entire lancing device surface and the inside of the cap with a Super Sani-Cloth (see 12 and 13).
- ③ For disinfecting the lancing device, use a new cloth. Repeat step 2 making sure that the surface stays wet for 2 minutes.
- ④ Wash hands thoroughly with soap and water.

Additionally Required Materials

CoaguChek® Softclix lancets or Accu-Chek® Softclix lancets

Discarding the Lancet and Lancing Device

Any product coming in contact with blood is considered contaminated (potentially infectious).* During obtaining a blood drop, any lancet and lancing device may come in contact with blood. Lancets and lancing devices may also be considered sharps. Disposal of sharps is regulated by law in many jurisdictions.

Roche is committed to recycling and sustainability. Comply with any laws or ordinances relating to the disposal of sharps and/or contaminated products. Contact your local health department or other appropriate authorities for proper handling and disposal of used lancets and used lancing devices.

Remove the lancet before discarding the lancing device.

*29 CFR 1910.1030 – Bloodborne pathogens

We Are Here to Help

If you have any questions about your lancing device or any procedure in these instructions, call CoaguChek Technical Service at 1-800-428-4674.

One (1) Year Warranty

Roche warrants the CoaguChek® XS

Softclix lancing device against defects in materials and workmanship for a period of one (1) year from the date of purchase. The liability of Roche under this warranty is limited to the repair or replacement, at Roche's option, of all parts or materials proven to be defective. This warranty does not cover repair or replacement of the CoaguChek® XS Softclix lancing devices which have been subjected to alteration, misuse, tampering, or abuse.

THE FOREGOING WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, WHETHER EXPRESS, IMPLIED, OR STATUTORY. ROCHE DISCLAIMS ANY AND ALL IMPLIED WARRANTIES, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE.

Roche's entire liability with respect to the CoaguChek® XS Softclix lancing device shall be the repair or replacement of nonconforming devices, at Roche's option. In no case shall Roche be liable for incidental, indirect, special, or consequential damages. This warranty gives you specific legal rights, and you may have other rights which vary from state to state so the above limitation and exclusion may not apply to you. All inquiries about the CoaguChek® XS Softclix lancing device under the above warranty or service policy must be directed to CoaguChek Technical Service by calling 1-800-428-4674. You will be informed of the procedure necessary for replacement. Please DO NOT return any device to Roche without authorization.

References

- ¹ FDA Public Health Notification: "Use of Fingertick Devices on More than one Person Poses Risk for Transmitting Bloodborne Pathogens: Initial Communication, (2010). Update 11/29/2010." <http://www.fda.gov/MedicalDevices/Safety/AlertsandNotices/ucm224025.htm>. Accessed June 1, 2016.
- ² CDC Clinical Reminder: "Use of Fingertick Devices on More than One Person Poses Risk for Transmitting Bloodborne Pathogens, (2010)." <http://www.cdc.gov/injectionsafety/Fingertick-DevicesBGM.html>. Accessed June 1, 2016.
- ³ Healthcare Infection Control Practices Advisory Committee (HICPAC), William A. Rutala, Ph.D., M.P.H., and David J. Weber, M.D., M.P.H. Centers for Disease Control and Prevention, 2008. "Guideline for Disinfection and Sterilization in Healthcare Facilities." Atlanta. http://www.cdc.gov/hicpac/pdf/guidelines/Disinfection_Nov_2008.pdf. Accessed June 1, 2016.

CoaguChek®

GTIN Global Trade Item Number

COAGUCHEK, SOFTCLIX, and ACCU-CHEK are trademarks of Roche.

©2017, Roche Diabetes Care

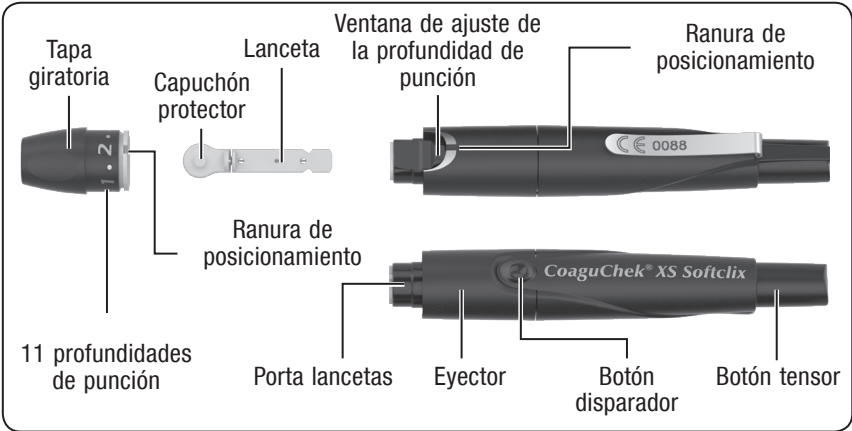
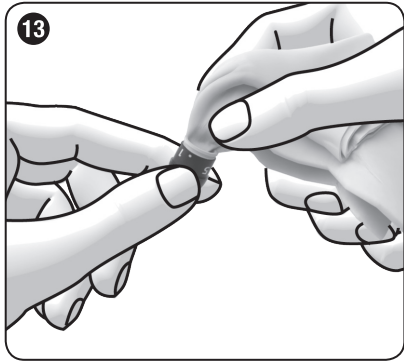
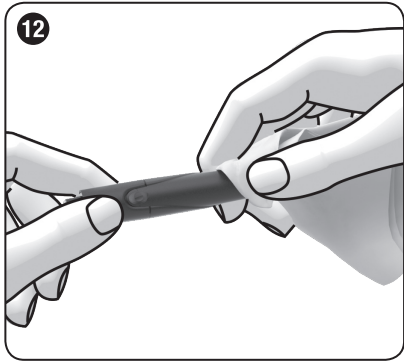
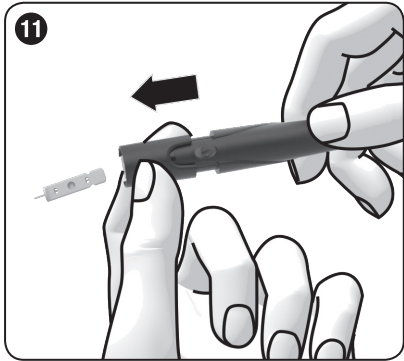
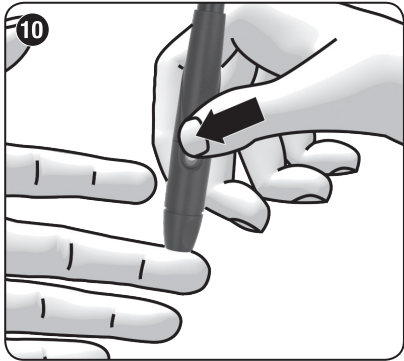
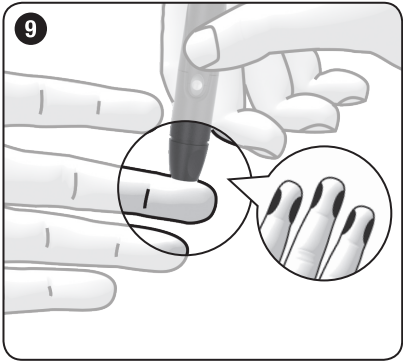
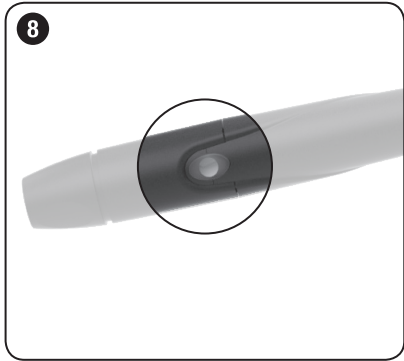
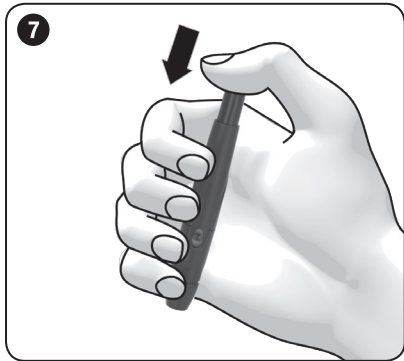
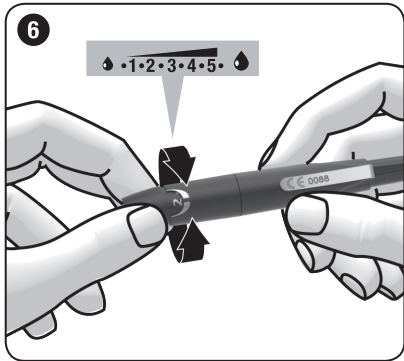
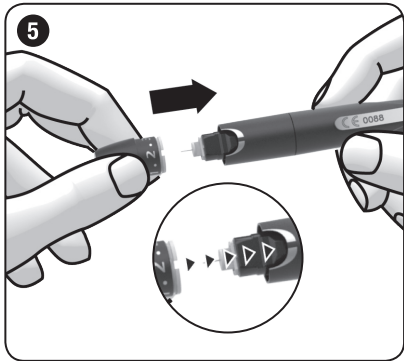
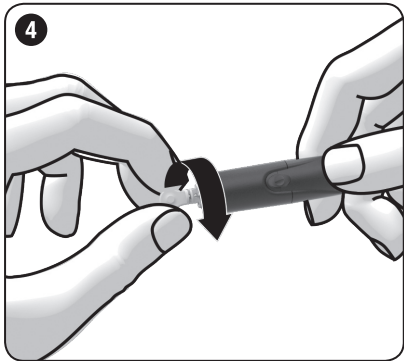
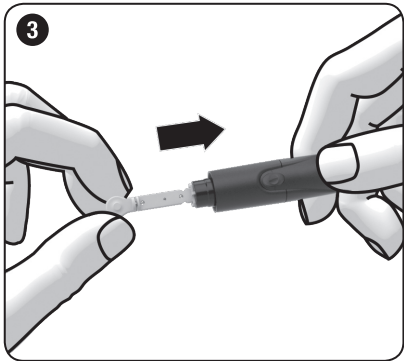
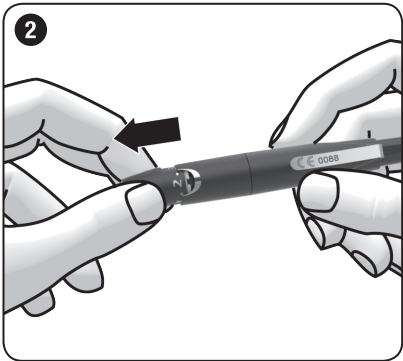
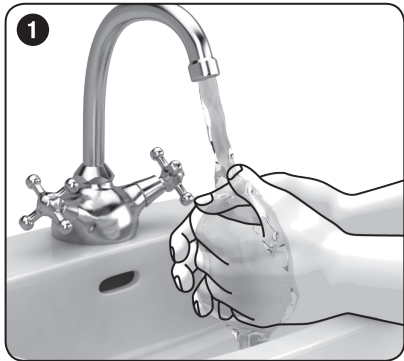
 Roche Diabetes Care GmbH
Sandhofer Strasse 116
D-68305 Mannheim
www.coaguheck-usa.com
www.roche.com

Distribution in USA by:
Roche Diagnostics, Indianapolis, IN

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References

- FDA Public Health Notification: "Use of Fingertick Devices on More than one Person Poses Risk for Transmitting Bloodborne Pathogens: Initial Communication, (2010). Update 11/29/2010." <http://www.fda.gov/MedicalDevices/Safety/AlertsandNotices/ucm224025.htm>. Accessed June 1, 2016.
- CDC Clinical Reminder: "Use of Fingertick Devices on More than One Person Poses Risk for Transmitting Bloodborne Pathogens, (2010)." <http://www.cdc.gov/injectionsafety/Fingertick-DevicesBGM.html>. Accessed June 1, 2016.
- Healthcare Infection Control Practices Advisory Committee (HICPAC), William A. Rutala, Ph.D., M.P.H., and David J. Weber, M.D., M.P.H. Centers for Disease Control and Prevention, 2008. "Guideline for Disinfection and Sterilization in Healthcare Facilities." Atlanta. http://www.cdc.gov/hicpac/pdf/guidelines/Disinfection_Nov_2008.pdf. Accessed June 1, 2016.

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