

REF	Ţ i	Σ	SYSTEM
09315284190*	00215284500	200	cobas e 402
09315284214*	09315284500	300	cobas e 801

^{*} Some kits shown may not be available in all countries.

English

System information

Short name	ACN (application code number)	Application
PBNPX	10237	18 minutes
PBNPSTX	10238	9 minutes (STAT = Short Turn Around Time)

Intended use

Immunoassay for the in vitro quantitative determination of N-terminal pro B-type natriuretic peptide in human serum and plasma. This assay is indicated as an aid in the diagnosis of individuals suspected of having congestive heart failure and detection of mild forms of cardiac dysfunction.1,2,3,4,5,6,7,8

The test also aids in the assessment of heart failure severity in patients diagnosed with congestive heart failure. 9,10

This assay is further indicated for the risk stratification of patients with acute coronary syndrome^{11,12,13,14,15} and congestive heart failure, and it can also be used for monitoring the treatment in patients with left ventricular dysfunction.^{1,2,16,17,18,19,20}

The test can help in the cardiovascular risk assessment of patients with type 2 diabetes mellitus. ^{21,22,23,24,25,26,27,28,29} The test is further indicated to aid in the identification of patients at risk with type 2 diabetes mellitus, without known history of cardiovascular disease, to optimize cardioprotective treatment. ³⁰

This test can be used to identify elderly individuals at high-risk for atrial fibrillation.³¹

The **e**lectro**c**hemiluminescence **i**mmuno**a**ssay "ECLIA" is intended for use on **cobas e** immunoassay analyzers.

Summary

Biology of natriuretic peptides

The significance of natriuretic peptides in the control of cardiovascular system function has been demonstrated. The following natriuretic peptides have been described: atrial natriuretic peptide (ANP), B-type natriuretic peptide (BNP), and C-type natriuretic peptide (CNP). 32,33

ANP and BNP, as antagonists of the renin-angiotensin-aldosterone system, influence by means of their natriuretic and diuretic properties, the electrolyte and fluid balance in an organism. 34,35,36

Definition of heart failure

Heart failure (HF) is a clinical syndrome characterized by systemic perfusion inadequate to meet the body's metabolic demands as a result of a structural and/or functional cardiac abnormality, resulting in a reduced cardiac output and/or elevated intracardiac pressures at rest or during stress. 1,2,3 Left ventricular dysfunction can be one of the functional precursors of HF. 1,2

HF is a progressive disease where in both hospitalized and ambulatory patients, most deaths are due to cardiovascular causes, mainly sudden death and worsening HF.^{1,2}

The typical terminology used to describe HF is based on measurement of the Left Ventricular Ejection Fraction (LVEF). According to latest ESC guidelines, HF comprises a wide range of patients, from those with normal LVEF [typically considered as \geq 50 %; HF with preserved EF (HFpEF)] to those with reduced LVEF [typically considered as < 40 %; HF with reduced EF (HFrEF)]. Patients with an LVEF in the range of 40-49 % represent a 'grey area', which is now defined as HF with midrange EF (HFmrEF). 1,2,3 Clinical information and imaging procedures are used to confirm the diagnosis of HF. 1,2,3

Based on the symptoms, the severity of HF is classified in stages (New York Heart Association classification [NYHA] I-IV). 1.2

HF symptoms are often non-specific and do not help to discriminate between HF and other conditions, such as (non-cardiogenic) pulmonary edema, chronic obstructive pulmonary disease (COPD), pneumonia or sepsis. 1.2

Clinical relevance of NT-proBNP in HF

In subjects with left ventricular dysfunction, serum and plasma concentrations of BNP increase, as does the concentration of the putatively inactive amino-terminal fragment, NT-proBNP. ProBNP, comprising 108 amino acids, is secreted mainly by the ventricle and, in this process, is cleaved into physiologically active BNP (77-108) and the N-terminal fragment NT-proBNP (1-76). 33,34

Several studies have demonstrated the significant role of natriuretic peptide testing, including NT-proBNP, in HF management from diagnosis to monitoring, leading to the recommendation to use them in clinical practice by major international guidelines with often highest level of evidence and recommendation. 1.2.3

NT-proBNP in the diagnosis of HF

The European Society of Cardiology HF Guidelines recommends natriuretic peptides, including NT-proBNP, as an initial diagnostic test.¹

Patients with NT-proBNP below the recommended NT-proBNP cutoffs for non-acute and acute onsets are unlikely to have HF, and therefore do not require echocardiography and elevated NT-proBNP levels help to identify patients who require further cardiac investigation.¹

When patients are grouped according to their NYHA classification, NT-proBNP levels increase with increasing NYHA class and reflect the severity of cardiac impairment.¹⁰

The test is also useful in the early stages of HF, where symptoms may be transient rather than present all the time.³ The high sensitivity of NT-proBNP allows also the detection of mild forms of cardiac dysfunction in asymptomatic patients with structural heart disease.^{4,5,6,7,8}

NT-proBNP in the management of HF in hospital setting

In patients hospitalized for acute decompensated HF, pre-discharge measurement of natriuretic peptides is useful to categorize patient's risk at discharge. $^{1.16}$ Changes in NT-proBNP levels during hospitalization demonstrated to be a strong predictor of outcomes. $^{16.37,38.39,40}$ A decrease in NT-proBNP values of ≥ 30 % has shown to be correlated with favorable outcome, while an increase in NT-proBNP values > 30 % was correlated with 6.6 times higher risk of rehospitalization or death in 6 months. 16

NT-proBNP in the management of HF in outpatient setting

In chronic heart failure (CHF), serial measurement of NT-proBNP concentration can be used to monitor the disease progression, to predict outcomes and evaluate the success of treatment. 1.2.17,18,20,41,42

Elevated NT-proBNP values are strongly predictive of adverse outcomes and rising values identify a risk, while significant lowering of NT-proBNP denotes improved outcomes and better prognosis. 1-2.17.43 When NT-proBNP levels change during treatment of chronic HF, decrease over the course of the disease correlates with improved clinical outcomes. 1-2.18,20 This interpretation of NT-proBNP results remains unchanged when using the new drug class Angiotensin receptor-neprilysin inhibitor 1-2 (ARNI, e.g. sacubitril-valsartan): In contrast to BNP, NT-proBNP degradation is not inhibited by the drug so that NT-proBNP results are not increased by the mode of action of the drug. 19,44,45 In patients treated with sacubitril-valsartan, rapid and sustained reduction of NT-proBNP levels has been observed, reflecting reduced wall stress³⁴ and benefits of the drug correlating with a lower rate of cardiovascular death and HF hospitalization. 20

NT-proBNP in patients with type 2 diabetes mellitus

Several clinical studies have consistently shown a graded relationship between circulating NT-proBNP concentration and cardiovascular risk: both single measurements and changes over time predict the occurrence of subsequent cardiovascular events.^{21,22,23,24,25,26,27,28,29,46,47}

In the PONTIAC study, 30 high-risk patients with type 2 diabetes mellitus but without known history of cardiovascular disease were identified by levels of NT-proBNP > 125 pg/mL. Those patients received an intensified cardiac



therapeutic strategy with up-titration of renin-angiotensin system antagonists and beta-blockers to the maximum tolerated dosages. Over 2 years of follow-up, this strategy led to a reduction of the rate of hospitalization or death due to cardiac disease by 65 %.

NT-proBNP in patients with atrial fibrillation

NT-proBNP values are an independent predictor of cardiovascular events including occurrence and recurrence of atrial fibrillation (AF).⁴⁸

NT-proBNP levels are highly associated with prevalent atrial fibrillation in the population aged 65 and more. $^{\rm 49,50}$

In the STROKESTOP II study, 31 all 75/76-year-old individuals residing in Stockholm region (n = 28712) were randomized in a 1:1 design to either be invited to a screening program for atrial fibrillation or to serve as a control group. Among the 6868 individuals who accepted the screening invitation, an index-ECG (30-s ECG) was recorded using a handheld one-lead device and NT-proBNP was analyzed in the 6315 participants without known atrial fibrillation. Participants were stratified into a low-risk group with NT-proBNP < 125 pg/mL and a high-risk group with NT-proBNP \geq 125 pg/mL. Participants in the high-risk group (n = 3766) with an index-ECG showing sinus rhythm were offered an extended ECG screening (2 week intermittent ambulatory handheld ECG recordings, four times daily). This stepwise mass-screening allowed to identify 4.4 % of the high-risk participants with unknown atrial fibrillation.

In addition, NT-proBNP has been included into the "ABC stroke risk score" taking into account age, biomarkers (cTnT hs and NT-proBNP) and history of prior stroke / transient ischemic attack. The ABC stroke risk score was shown to significantly improve the prediction of stroke in AF patients compared to the widely used CHA2DS2-VASc score.⁵¹ Results of the ENGAGE AF TIMI 48 trial evaluating the ABC stroke and the ABC bleeding risk scores (taking into account age, biomarkers GDF-15, cTnT-hs, and hemoglobin, and history of bleeding) confirmed that these scores may help to identify AF patients most likely to benefit from treatment with non-vitamin K antagonist oral anticoagulants (NOACs).⁵²

NT-proBNP in other populations at risk of CVD/HF

NT-proBNP can also be used for prognostic applications in patients with acute coronary syndrome. The GUSTO IV study, with more than 6800 patients, showed that NT-proBNP was the strongest independent predictor of one year mortality in patients with acute coronary syndrome. ¹⁵

NT-proBNP can be used to identify patients at higher cardiovascular risk receiving non-cardiovascular treatments. It may be helpful in monitoring the use and dosing of cancer drugs⁵³ or interventions causing fluid retention or volume overload (e.g. COX-2 inhibitors, nonsteroidal anti inflammatory drugs).^{54,55,56,57,58} NT-proBNP can be used before non-cardiac surgery to evaluate patients' perioperative cardiac risk.⁵⁹

In meta-analysis including 95617 patients without history of cardiovascular disease, NT-proBNP concentration strongly predicted first-onset HF and augmented chronic heart disease and stroke prediction, suggesting that NT-proBNP could serve as a biomarker in new therapeutic approaches that integrate HF into cardiovascular disease primary prevention. 60

The Elecsys proBNP II assay contains two monoclonal antibodies which recognize epitopes located in the N-terminal part (1-76) of proBNP (1-108).

Test principle

Sandwich principle.

Total duration of assay: 18 minutes.

- 1st incubation: Antigen in the sample (9 µL), a biotinylated monoclonal NT-proBNP-specific antibody, and a monoclonal NT-proBNP-specific antibody labeled with a ruthenium complex^{a)} form a sandwich complex.
- 2nd incubation: After addition of streptavidin-coated microparticles, the complex becomes bound to the solid phase via interaction of biotin and streptavidin.

Total duration of assay: 9 minutes.

 During a 9 minute incubation, antigen in the sample (9 µL), a biotinylated monoclonal NT-proBNP-specific antibody, a monoclonal NT-proBNP-specific antibody labeled with a ruthenium complex and streptavidin-coated microparticles react to form a sandwich complex, which is bound to the solid phase.

For both assay applications:

- The reaction mixture is aspirated into the measuring cell where the microparticles are magnetically captured onto the surface of the electrode. Unbound substances are then removed with ProCell II M. Application of a voltage to the electrode then induces chemiluminescent emission which is measured by a photomultiplier.
- Results are determined via a calibration curve which is instrumentspecifically generated by 2-point calibration and a master curve provided via the cobas link.

a) Tris(2,2'-bipyridyl)ruthenium(II)-complex (Ru(bpy)3+)

Reagents - working solutions

The cobas e pack is labeled as PBNPX.

- M Streptavidin-coated microparticles, 1 bottle, 12.4 mL: Streptavidin-coated microparticles 0.72 mg/mL; preservative.
- R1 Anti-NT-proBNP-Ab~biotin, 1 bottle, 21.0 mL: Biotinylated monoclonal anti-NT-proBNP antibody (mouse) 1.1 μg/mL; phosphate buffer 40 mmol/L, pH 5.8; preservative.
- R2 Anti-NT-proBNP-Ab~Ru(bpy)²⁺₃, 1 bottle, 19.7 mL: Monoclonal anti-NT-proBNP antibody (sheep) labeled with ruthenium complex 1.1 μg/mL; phosphate buffer 40 mmol/L, pH 5.8; preservative.

Precautions and warnings

For in vitro diagnostic use for health care professionals. Exercise the normal precautions required for handling all laboratory reagents.

Infectious or microbial waste:

Warning: handle waste as potentially biohazardous material. Dispose of waste according to accepted laboratory instructions and procedures.

Environmental hazards:

Apply all relevant local disposal regulations to determine the safe disposal. Safety data sheet available for professional user on request.

This kit contains components classified as follows in accordance with the Regulation (EC) No. 1272/2008:



Warning

H317 May cause an allergic skin reaction.

Prevention:

P261 Avoid breathing mist or vapours.

P272 Contaminated work clothing should not be allowed out of

the workplace.

P280 Wear protective gloves.

Response:

P333 + P313 If skin irritation or rash occurs: Get medical

advice/attention.

P362 + P364 Take off contaminated clothing and wash it before reuse.

Disposal:

P501 Dispose of contents/container to an approved waste

disposal plant.

Product safety labeling follows EU GHS guidance.

Contact phone: all countries: +49-621-7590

Avoid foam formation in all reagents and sample types (specimens, calibrators and controls).

Reagent handling

The Elecsys proBNP II assay can be used for both the 9-minute application and the 18-minute application.



The reagents in the kit have been assembled into a ready-for-use unit that cannot be separated.

All information required for correct operation is available via the cobas link.

Storage and stability

Store at 2-8 °C.

Do not freeze.

Store the **cobas e** pack **upright** in order to ensure complete availability of the microparticles during automatic mixing prior to use.

Stability:	
unopened at 2-8 °C	up to the stated expiration date
on the analyzers	16 weeks

Specimen collection and preparation

Only the specimens listed below were tested and found acceptable. Serum collected using standard sampling tubes or tubes containing separating gel.

Li-heparin, K2-EDTA and K3-EDTA plasma.

Plasma tubes containing separating gel can be used.

Criterion: Slope 0.9-1.1 + intercept within \leq ± 10 pg/mL + coefficient of correlation \geq 0.95.

Stable for 3 days at 20-25 °C, 6 days at 2-8 °C, 24 months at -20 °C (\pm 5 °C). Freeze only once.

The sample types listed were tested with a selection of sample collection tubes that were commercially available at the time of testing, i.e. not all available tubes of all manufacturers were tested. Sample collection systems from various manufacturers may contain differing materials which could affect the test results in some cases. When processing samples in primary tubes (sample collection systems), follow the instructions of the tube

Centrifuge samples containing precipitates before performing the assay. Do not use samples and controls stabilized with azide.

Ensure the samples and calibrators are at 20-25 °C prior to measurement.

Due to possible evaporation effects, samples and calibrators on the analyzers should be analyzed/measured within 2 hours.

Materials provided

See "Reagents – working solutions" section for reagents.

Materials required (but not provided)

- REF 09315314190, CalSet proBNP II, for 4 x 1.0 mL
- REF 04917049190, PreciControl Cardiac II, for 2 x 2.0 mL
- REF 07299001190, Diluent Universal, 36 mL sample diluent
- General laboratory equipment
- cobas e analyzer

Additional materials for cobas e 402 and cobas e 801 analyzers:

- REF 06908799190, ProCell II M, 2 x 2 L system solution
- REF 04880293190, CleanCell M, 2 x 2 L measuring cell cleaning solution
- REF 07485409001, Reservoir Cup, 8 cups to supply ProCell II M and CleanCell M
- REF 06908853190, PreClean II M, 2 x 2 L wash solution
- REF 05694302001, Assay Tip/Assay Cup tray, 6 magazines
 x 6 magazine stacks x 105 assay tips and 105 assay cups, 3 wasteliners
- REF 07485425001, Liquid Flow Cleaning Cup, 2 adaptor cups to supply ISE Cleaning Solution/Elecsys SysClean for Liquid Flow Cleaning Detection Unit
- REF 07485433001, PreWash Liquid Flow Cleaning Cup, 1 adaptor cup to supply ISE Cleaning Solution/Elecsys SysClean for Liquid Flow Cleaning PreWash Unit
- REF 11298500316, ISE Cleaning Solution/Elecsys SysClean, 5 x 100 mL system cleaning solution

Assay

For optimum performance of the assay follow the directions given in this document for the analyzer concerned. Refer to the appropriate operator's manual for analyzer-specific assay instructions.

Resuspension of the microparticles takes place automatically prior to use.

Place the cooled (stored at 2-8 °C) **cobas e** pack on the reagent manager. Avoid foam formation. The system automatically regulates the temperature of the reagents and the opening/closing of the **cobas e** pack.

Calibration

Traceability: This method has been standardized against the Elecsys proBNP assay ([REF] 03121640122). This in turn is traceable to pure synthetic NT-proBNP (1-76) by weight.

The predefined master curve is adapted to the analyzer using the relevant CalSet.

Calibration frequency: Calibration must be performed once per reagent lot using fresh reagent (i.e. not more than 24 hours since the **cobas e** pack was registered on the analyzer).

Calibration interval may be extended based on acceptable verification of calibration by the laboratory.

Renewed calibration is recommended as follows:

- after 12 weeks when using the same reagent lot
- after 28 days when using the same cobas e pack on the analyzer
- as required: e.g. quality control findings outside the defined limits

Quality control

For quality control, use PreciControl Cardiac II.

In addition, other suitable control material can be used.

Controls for the various concentration ranges should be run individually at least once every 24 hours when the test is in use, once per **cobas e** pack, and following each calibration.

The control intervals and limits should be adapted to each laboratory's individual requirements. Values obtained should fall within the defined limits. Each laboratory should establish corrective measures to be taken if values fall outside the defined limits.

If necessary, repeat the measurement of the samples concerned.

Follow the applicable government regulations and local guidelines for quality control.

Calculation

The analyzer automatically calculates the analyte concentration of each sample (either in pmol/L or pg/mL).

Conversion factors: $pmol/L \times 8.457 = pg/mL$ $pg/mL \times 0.118 = pmol/L$

Limitations - interference

The effect of the following endogenous substances and pharmaceutical compounds on assay performance was tested. Interferences were tested up to the listed concentrations and no impact on results was observed.

Endogenous substances

Compound	Concentration tested
Bilirubin	≤ 428 μmol/L or ≤ 25 mg/dL
Hemoglobin	≤ 0.621 mmol/L or ≤ 1000 mg/dL
Intralipid	≤ 1500 mg/dL
Biotin	≤ 14326 nmol/L or ≤ 3500 ng/mL
Rheumatoid factors	≤ 1500 IU/mL
IgG	≤ 6.0 g/dL
IgA	≤ 1.6 g/dL
IgM	≤ 1.0 g/dL

Criterion: Recovery of \pm 10 pg/mL of initial value \leq 100 pg/mL and \pm 10 % of initial value > 100 pg/mL.

There is no high-dose hook effect at NT-proBNP concentrations up to 35400 pmol/L (300000 pg/mL).



Pharmaceutical substances

In vitro tests were performed on 16 commonly used pharmaceuticals. No interference with the assay was found.

In addition, the following special cardiac drugs were tested. No interference with the assay was found.

Special cardiac drugs

Drug	Concentration tested mg/L
Carvedilol	37.5
Clopidogrel	75.0
Digoxin	0.25
Epinephrine (Adrenaline)	0.50
Insulin	1.60
Lidocaine	80.0
Lisinopril	10.0
Methylprednisolone	7.50
Metoprolol	150
Nifedipine	30.0
Phenprocoumon (Marcumar)	3.00
Propafenone	300
Reteplase	33.3
Simvastatin	30.0
Spironolactone	75.0
Tolbutamide	1500
Torasemide	15.0
Verapamil	240

In rare cases, interference due to extremely high titers of antibodies to analyte-specific antibodies, streptavidin or ruthenium can occur. These effects are minimized by suitable test design.

In extremely rare cases (global incidence: < 1 in 10 million), patients may show discrepant results when tested with the assay kit (values < Limit of Detection) due to a NT-proBNP genetic variant.

For diagnostic purposes, the results should always be assessed in conjunction with the patient's medical history, clinical examination and other findings.

Limits and ranges Measuring range

5-35000 pg/mL or 0.6-4130 pmol/L (defined by the Limit of Detection and the maximum of the master curve). Values below the Limit of Detection are reported as < 5 pg/mL (< 0.6 pmol/L). Values above the measuring range are reported as > 35000 pg/mL (> 4130 pmol/L) or up to 70000 pg/mL (8260 pmol/L) for 2-fold diluted samples.

Lower limits of measurement

Limit of Blank, Limit of Detection and Limit of Quantitation

Limit of Blank = 3 pg/mL (0.4 pmol/L)

Limit of Detection = 5 pg/mL (0.6 pmol/L)

Limit of Quantitation = 50 pg/mL (5.9 pmol/L)

The Limit of Blank, Limit of Detection and Limit of Quantitation were determined in accordance with the CLSI (Clinical and Laboratory Standards Institute) EP17-A2 requirements.

The Limit of Blank is the 95^{th} percentile value from $n \ge 60$ measurements of analyte-free samples over several independent series. The Limit of Blank corresponds to the concentration below which analyte-free samples are found with a probability of 95%.

The Limit of Detection is determined based on the Limit of Blank and the standard deviation of low concentration samples. The Limit of Detection corresponds to the lowest analyte concentration which can be detected (value above the Limit of Blank with a probability of 95 %).

The Limit of Quantitation is the lowest analyte concentration that can be reproducibly measured with an intermediate precision CV of \leq 20 %.

Dilution

Samples with NT-proBNP concentrations above the measuring range can be diluted with Diluent Universal. The recommended dilution is 1:2 (either automatically by the analyzers or manually). The concentration of the diluted sample must be \geq 1770 pmol/L or \geq 15000 pg/mL.

After manual dilution, multiply the result by the dilution factor.

After dilution by the analyzers, the software automatically takes the dilution into account when calculating the sample concentration.

Dilutions of up to 1:10 may entail maximum deviations of 25 % from the theoretical value.

Clinical data

Interpretation of NT-proBNP values

With increasing age atherosclerosis and aging processes of the heart (e.g. fibrosis) result in cardiac dysfunction. Development of cardiac dysfunction is individually different and clinically asymptomatic in its early stages. ^{61,62} NT-proBNP levels reflect cardiac function or dysfunction respectively. With increasing age elevated levels of NT-proBNP are more frequently found in apparently healthy individuals, thus reflecting the increasing frequency of cardiac dysfunction.

NT-proBNP values need to be interpreted in conjunction with the medical history, clinical findings and other information (e.g. imaging, laboratory findings, accompanying disorders, treatment effects).

Expected values

NT-proBNP concentrations in the reference group are shown in the following tables; they reflect NT-proBNP levels in individuals without known cardiovascular disease and should not be used for the differential diagnosis of HF

Each laboratory should investigate the transferability of the expected values to its own patient population and if necessary determine its own reference ranges

Reference group

The circulating NT-proBNP concentration was determined in samples from 4266 subjects aged between 35 and 74 years, enrolled into the Gutenberg Health Study in Germany. 63 These individuals had no prevalent cardiovascular diseases such as former history of stroke, myocardial infarction, coronary artery disease, peripheral artery disease, chronic heart failure or atrial fibrillation. The descriptive statistics for NT-proBNP (pg/mL) in the reference group are shown in the following table:

Age (years)	Men				Won	nen		
	Median	95 th	97.5 th	99 th	Median	95 th	97.5 th	99 th
		per-	per-	per-		per-	per-	per-
		cent-	cent-	cent-		cent-	cent-	cent-
		ile	ile	ile		ile	ile	ile
35-44	18.9	90.8	115	137	59.9	202	237	311
45-54	23.5	121	173	273	63.8	226	284	395
55-64	47.4	262	386	920	81.8	284	352	417
65-74	89.3	486	879	2346	133	470	623	784
All	35.6	238	344	703	78.6	304	389	509

The circulating NT-proBNP concentration was also determined in samples from 2812 subjects aged between 20 and above 70 years, enrolled in a cardiovascular health screening program at a tertiary medical center in Taipei, Taiwan.⁶⁴ These individuals had no known cardiovascular or systemic co-morbidities, and no structural heart diseases. The descriptive statistics for NT-proBNP (pg/mL) in the reference group are shown in the following table:



Age (years)	Men (N = 1836)				Women	(N = 976	5)	
	N	Median	25 th	75 th	N	Median	25 th	75 th
			per- centile	per- centile			per- centile	per- centile
20-29	48	9	5.0	19.7	33	30.1	10.3	41.9
30-39	369	13.5	5.0	29.7	153	34.9	20.8	60.4
40-49	708	17	7.8	32.4	346	40.1	18.9	62.5
50-59	558	22.8	11.6	42.6	310	44.4	27.3	64.7
60-69	130	31.5	16.6	59.1	112	61.7	30.8	85.2
≥ 70	23	66.1	34.2	106.6	22	77.5	46.3	123.0

In the pediatric population aged between 1 and 18 the following NT-proBNP values were obtained using the Elecsys proBNP assay,

[REF] 03121640122:⁵⁵

Age (years)	N	NT-proBNP (ng/L)				
		75 th percentile	97.5 th percentile			
1-3	13	231	320			
4-6	21	113	190			
7-9	32	94	145			
10	11	73	112			
11	69	93	317			
12	21	95	186			
13	23	114	370			
14	18	68 363				
15	24	74 217				
16	24	85 206				
17	24	71 135				
18	12	53	115			

Recommended cutoffs in patients for diagnosis of chronic heart failure in non-acute onset

A number of studies and ESC guidelines support a decision threshold for NT-proBNP of 125 pg/mL in non-acute onset for the diagnosis of HF.¹,3,66,67,68,69,70 NT-proBNP values < 125 pg/mL exclude cardiac dysfunction with a high level of certainty in patients with symptoms suggestive of HF e.g. dyspnea. NT-proBNP values ≥ 125 pg/mL may indicate cardiac dysfunction and are associated with an increased risk of cardiac complications (myocardial infarction, HF, death). At the cut-off value, ESC Guidelines state that natriuretic peptides have a very high negative predictive value (NPV) comprised between 94 % and 98 % and a positive predictive value (PPV) comprised between 44 % and 57 %.¹

Patients with stable HF (n = 721) including patients with asymptomatic left ventricular dysfunction (n = 176) and patients with congestive heart failure (n = 545) were compared to a reference group (n = 2264).

ROC plot analysis at the cutoff value of 125 pg/mL showed a sensitivity of 90 % and a specificity of 93 %.

Correlation of NT-proBNP with NYHA classification in patients diagnosed with chronic heart failure

NT-proBNP values (pg/mL) for patients with reduced left ventricular ejection fraction (majority under therapy):


NYHA functional class								
	NYHA I NYHA II NYHA III NYHA IV							
N	182	250	234	35				
Mean	1016	1666	3029	3465				
SD	1951	2035	4600	4453				
Median	342	951	1571	1707				
5 th percentile	32.9	103	126	148				

NYHA functional class						
NYHA I NYHA II NYHA III NYHA IV						
95 th percentile 3410 6567 10449 12188						
% > 125 pg/mL	78.6	94.0	95.3	97.1		

Recommended cutoffs in patients for diagnosis of chronic heart failure in acute onset

ICON (International Collaborative of NT-proBNP) study¹⁰

NT-proBNP concentrations were determined in samples from 1256 patients presenting with acute shortness of breath to emergency departments at four hospitals. This population included patients with a prior history of hypertension, coronary artery disease, myocardial infarction, HF, or pulmonary disease. 720 subjects were found to be suffering from acute exacerbation of HF, while the remainder were determined to present dyspnea due to other causes. The descriptive statistics for NT-proBNP concentrations (pg/mL) for both groups are shown in the following figure adapted from the ICON study:¹⁰

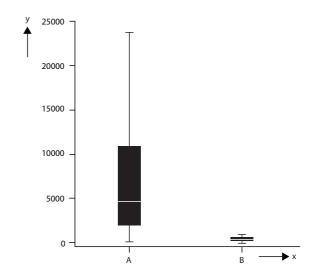

 $X \rightarrow A$: Acute HF (n = 720); B: Not acute HF (n = 536) $Y \rightarrow NT$ -proBNP (pg/mL)

Diagnostic category	Median (IQR) NT-proBNP, pg/mL
Acute HF	4639 (1882-10818)
Not Acute HF	108 (37-381)

By using the optimal cutoffs established by the ICON study group and shown in the table below, physicians can increase the specificity and accuracy for diagnosing HF in patients presenting acute dyspnea in the emergent setting.

Category	Optimal cut-point pg/mL	Sensitivity %	Specificity %	PPV %	NPV %	Accuracy %
Rule in cut-	point					
< 50 years (n = 184)	450	97	93	76	99	94
50-75 years (n = 537)	900	90	82	83	88	85



Category	Optimal cut-point pg/mL	Sensitivity %	Specificity %	PPV %	NPV %	Accuracy %
> 75 years (n = 535)	1800	85	73	92	55	83
Rule out cut-point						
All patients (n = 1256)	300	99	60	77	98	83

NT-proBNP concentrations were determined in samples from patients presenting with acute shortness of breath to emergency departments in Singapore (n = 606) and in New Zealand (n = 500). This population included patients with a prior history of hypertension, hyperlipidemia, coronary artery disease, myocardial infarction, HF, or pulmonary disease. NT-proBNP concentration in patients with final adjudicated diagnosis of acute heart failure was 4234 [2008-9799] pg/mL in Singapore (median [25-75th percentile], n = 148) and 4429 [2123-9479] pg/mL in New Zealand (n = 180).

The diagnostic performances of NT-proBNP at the cutoffs established in the ICON Study¹⁰ are shown in the table below for both populations:

Category	Optimal cut-point pg/mL	Sensitivity %	Specificity %	PPV %	NPV %	Accuracy %		
Rule in cut-	-point	1				,		
< 50 years	< 50 years							
Singapore (n = 196)	450	100	91	58	100	92		
New Zealand (n = 41)		86	76	43	96	78		
50-75 years	3	•	•		•			
Singapore (n=350)	900	88	83	68	95	85		
New Zealand (n = 236)		91	75	58	96	80		
>75 years								
Singapore (n = 60)	1800	79	81	73	85	80		
New Zealand (n = 223)		87	63	69	84	75		
Rule out cu	Rule out cut-point							
All patients								
Singapore (n = 606)	300	97	73	54	99	79		
New Zealand (n = 500)		97	42	49	96	62		

Specific performance data

Representative performance data on the analyzers are given below. Results obtained in individual laboratories may differ.

Precision

Precision was determined using Elecsys reagents, pooled human sera and controls in a protocol (EP05-A3) of the CLSI (Clinical and Laboratory Standards Institute): 2 runs per day in duplicate each for 21 days (n = 84). The following results were obtained:

cobas e 402 and cobas e 801 analyzers (18-minute application)						
	Repeatability					
Sample	Me	Mean		SD		
	pg/mL	pmol/L	pg/mL	pmol/L	%	
Human serum 1	20.2	2.38	1.87	0.221	9.2	
Human serum 2	67.7	7.99	1.86	0.219	2.8	
Human serum 3	145	17.1	2.84	0.335	2.0	
Human serum 4	462	54.5	9.87	1.16	2.1	
Human serum 5	1014	120	17.2	2.03	1.7	
Human serum 6	2088	246	42.8	5.05	2.0	
Human serum 7	16399	1935	274	32.3	1.7	
Human serum 8	33786	3987	731	86.3	2.2	
PC CARDIIb)1	140	16.5	2.11	0.249	1.5	
PC CARDII2	4846	572	89.3	10.5	1.8	

b) PC CARDII = PreciControl Cardiac II

cobas e 402 and cobas e 801 analyzers (18-minute application)						
	Intermediate precision					
Sample	Mean		S	CV		
	pg/mL	pmol/L	pg/mL	pmol/L	%	
Human serum 1	20.2	2.38	2.52	0.297	12.5	
Human serum 2	67.7	7.99	2.69	0.317	4.0	
Human serum 3	145	17.1	4.86	0.573	3.4	
Human serum 4	462	54.5	14.8	1.75	3.2	
Human serum 5	1014	120	30.9	3.65	3.0	
Human serum 6	2088	246	73.2	8.64	3.5	
Human serum 7	16399	1935	553	65.3	3.4	
Human serum 8	33786	3987	1153	136	3.4	
PC CARDII1	140	16.5	4.07	0.480	2.9	
PC CARDII2	4846	572	141	16.6	2.9	

cobas e 402 and cobas e 801 analyzers (9-minute application)						
	Repeatability					
Sample	Mean		S	CV		
	pg/mL	pmol/L	pg/mL	pmol/L	%	
Human serum 1	20.8	2.45	0.677	0.080	3.3	
Human serum 2	65.1	7.68	1.51	0.178	2.3	
Human serum 3	138	16.3	2.36	0.278	1.7	
Human serum 4	443	52.3	8.74	1.03	2.0	
Human serum 5	970	114	17.0	2.01	1.7	
Human serum 6	2012	237	33.4	3.94	1.7	
Human serum 7	17514	2067	261	30.8	1.5	
Human serum 8	33353	3936	679	80.1	2.0	
PC CARDII1	148	17.5	2.20	0.260	1.5	
PC CARDII2	5229	617	85.4	10.1	1.6	



cobas e 402 and cobas e 801 analyzers (9-minute application)						
	Intermediate precision					
Sample	Me	ean	SD		CV	
	pg/mL	pmol/L	pg/mL	pmol/L	%	
Human serum 1	20.8	2.45	0.961	0.113	4.6	
Human serum 2	65.1	7.68	2.09	0.247	3.2	
Human serum 3	138	16.3	4.27	0.504	3.1	
Human serum 4	443	52.3	13.0	1.53	2.9	
Human serum 5	970	114	25.0	2.95	2.6	
Human serum 6	2012	237	54.6	6.44	2.7	
Human serum 7	17514	2067	565	66.7	3.2	
Human serum 8	33353	3936	939	111	2.8	
PC CARDII1	148	17.5	3.89	0.459	2.6	
PC CARDII2	5229	617	137	16.2	2.6	

Method comparison

a) A comparison of the Elecsys proBNP II assay (18-minute application; y), \$| REF | 08836752190 / 09315284190 | with the Elecsys proBNP II assay (18-minute application; x), \$| REF | 07027664190 | on the **cobas e** 801 analyzer gave the following correlations (pg/mL):

Number of samples measured: 163

Passing/Bablok⁷² Linear regression y = 1.02x + 1.66 y = 1.02x - 3.92 r = 0.991 r = 1.00

The sample concentrations were between 16.8 and 34019 pg/mL (1.98 and 4014 pmol/L).

b) A comparison of the Elecsys proBNP II assay (9-minute application; y), [REF] 08836752190 / 09315284190 with the Elecsys proBNP II assay (9-minute application; x), [REF] 07027664190 on the **cobas e** 801 analyzer gave the following correlations (pg/mL):

Number of samples measured: 162

Passing/Bablok⁷² Linear regression y = 1.02x - 7.82 y = 1.01x + 64.1 r = 0.990 r = 1.00

The sample concentrations were between 16.5 and 34207 pg/mL (1.95 and 4036 pmol/L).

c) A comparison of the Elecsys proBNP II assay (18-minute application), REF 08836752190 / 09315284190 (**cobas e** 801 analyzer; y), with the Elecsys proBNP II assay, REF 08836736190 / 09315268190 (**cobas e** 601 analyzer; x), gave the following correlations (pg/mL):

Number of samples measured: 162

 $\begin{array}{ll} Passing/Bablok^{72} & Linear \ regression \\ y = 1.03x - 3.43 & y = 1.04x - 16.3 \\ \tau = 0.985 & r = 0.999 \end{array}$

The sample concentrations were between 16.8 and 34019 pg/mL (1.98 and 4014 pmol/L).

d) A comparison of the Elecsys proBNP II assay (9-minute application; y), REF 08836752190 / 09315284190 with the Elecsys proBNP II assay (18-minute application; x), REF 08836752190 / 09315284190 on the **cobas e** 801 analyzer gave the following correlations (pg/mL):

Number of samples measured: 161

Passing/Bablok⁷² Linear regression y = 0.973x + 1.02 y = 0.960x + 56.2 t = 0.991 t = 0.999

The sample concentrations were between 16.5 and 33783 pg/mL (1.95 and 3986 pmol/L).

e) A comparison of the Elecsys proBNP II assay, [REF] 08836752190 / 09315284190 (**cobas e** 402 analyzer; y) with the Elecsys proBNP II assay, [REF] 08836752190 / 09315284190 (**cobas e** 801 analyzer; x) gave the following correlations (pg/mL):

Number of samples measured: 176

 $\begin{array}{ll} \mbox{Passing/Bablok}^{72} & \mbox{Linear regression} \\ \mbox{y} = 0.956 \mbox{x} - 5.70 & \mbox{y} = 0.954 \mbox{x} + 37.6 \\ \mbox{T} = 0.964 & \mbox{r} = 0.997 \\ \end{array}$

The sample concentrations were between 11.0 and 34847 pg/mL (1.30 and 4112 pmol/L).

Analytical specificity

The Elecsys proBNP II assay does not show any significant cross-reactivity with the following substances, tested with NT-proBNP concentrations of approximately 100 pg/mL and 2500 pg/mL (maximum tested concentration):

Cross-reactant	Concentration tested
Adrenomedullin	1.0 ng/mL
Aldosterone	0.6 ng/mL
Angiotensin I	0.6 ng/mL
Angiotensin II	0.6 ng/mL
Angiotensin III	1.0 ng/mL
ANP ₂₈	3.1 μg/mL
Arg-vasopressin	1.0 ng/mL
BNP ₃₂	3.5 μg/mL
CNP ₂₂	2.2 μg/mL
Endothelin	20 pg/mL
NT-proANP ₁₋₃₀ (preproANP ₂₆₋₅₅)	3.5 μg/mL
NT-proANP ₃₁₋₆₇ (preproANP ₅₆₋₉₂)	1.0 ng/mL
NT-proANP ₇₉₋₉₈ (preproANP ₁₀₄₋₁₂₃)	1.0 ng/mL
Renin	50 ng/mL
Urodilatin	3.5 μg/mL

References

- 1 Ponikowski P, Voors AA, Anker SD, et al. 2016 ESC Guidelines for the diagnosis and treatment of acute and chronic heart failure. Eur J Heart Fail 2016;18(8):891-975.
- Yancy CW, Jessup M, Bozkurt B, et al. 2013 ACCF/AHA guideline for the management of heart failure: a report of the American College of Cardiology. Foundation/American Heart Association Task Force on practice guidelines. Circulation 2013 15;128(16):1810-1852.
- 3 Taylor CJ, Rutten FH, Brouwer JR, et al. Practical guidance on heart failure diagnosis and management in primary care: recent EPCCS recommendations. Br J Gen Pract. 2017;67(660):326-327.
- 4 Mueller T, Gegenhuber A, Poelz W, et al. Head-to-head comparison of the diagnostic utility of BNP and NT-proBNP in symptomatic and asymptomatic structural heart disease. Clin Chim Acta 2004;341:41-48.
- McGrady M, Reid CM, Shiel L, et al. NT-proB natriuretic peptide, risk factors and asymptomatic left ventricular dysfunction: results of the SCReening Evaluation of the Evolution of New Heart Failure study (SCREEN-HF). Int J of Card 2013;169(2):133-138.
- 6 Costello-Boerrigter LC, Boerrigter G, Redfield MM, et al. Aminoterminal pro-B-type natriuretic peptide and B-type natriuretic peptide in the general community: determinants and detection of left ventricular dysfunction. J Am Coll Cardiol 2006;47(2):345-353.
- O'Donoghue M, Chen A, Baggish AL, et al. The effects of ejection fraction on N-terminal ProBNP and BNP levels in patients with acute CHF: analysis from the ProBNP Investigation of Dyspnea in the Emergency Department (PRIDE) study. Journ of Card Fail 2005;11(5):9-14.



- 8 Mureddu GF, Tarantini L, Agabiti N, et al. Evaluation of different strategies for identifying asymptomatic left ventricular dysfunction and pre-clinical (stage B) heart failure in the elderly. Results from 'PREDICTOR', a population based-study in central Italy European Journal of Heart Failure 2013;15:1102-1112.
- 9 Hunt PJ, Richards AM, Nicholls MG, et al. Immunoreactive amino terminal pro-brain natriuretic peptide (NT-PROBNP): a new marker of cardiac impairment. Clin Endocrinol 1997;47(3):287-296.
- Januzzi JL, van Kimmenade R, Lainchbury J, et al. NT-proBNP testing for diagnosis and short-term prognosis in acute destabilized heart failure: an international pooled analysis of 1256 patients The International Collaborative of NT-proBNP Study Euro Heart Journ 2006;27(3):330-337.
- 11 Richards AM, Nicholls GM, Yandle TG, et al. Plasma N-Terminal Pro-Brain Natriuretic Peptide and Adrenomedullin: New Neurohormonal Predictors of Left Ventricular Function and Prognosis After Myocardial Infarction. Circulation 1998;97:1921-1929.
- 12 Galvani M, Ferrini D, Ottani F. Natriuretic peptides for risk stratification of patients with acute coronary sydromes. Eur J Heart Fail 2004;6:327-333.
- 13 Nørgaard BL, Terkelsen CJ, Riiskjaer M, et al. Risk prediction in acute coronary syndrome from serial inhospital measurements of N-terminal pro-B-type natriuretic peptide. Acute Card Care 2008;10:159-166.
- 14 Dallmeier D, Pencina MJ, Rajman I, et al. Serial measurements of Nterminal pro-brain natriuretic peptide in patients with coronary heart disease. PLoS One 2015;28;10(1):e0117143.
- James SK, Lindahl B, Siegbahn A, et al. NT proBNP and other Risk Markers for the Separate Prediction of Mortality and Subsequent Myocardial Infarction in Patients with Unstable Coronary Artery Disease. GUSTO IV Substudy. Circulation 2003;108:275-281.
- 16 Bettencourt P, Azevedo A, Pimenta J, et al. N-Terminal-Pro-Brain Natriuretic Peptide Predicts Outcome After Hospital Discharge in. Circulation 2004;110(15):2168-2174.
- 17 Masson S, Latini R, Anand IS, et al. Prognostic value of changes in Nterminal pro-brain natriuretic peptide in Val-HeFT (Valsartan Heart Failure Trial). J Am Coll Cardiol 2008;16;52(12):997-1003.
- 18 Januzzi JL, Throughton R. Are Serial BNP Measurements Useful in Heart Failure Management? Serial Natriuretic Peptide Measurements Are Useful in Heart Failure Management. Circulation 2013;127:500-508.
- 19 Mair J, Lindahl B, Giannitsis E, et al. Will sacubitril-valsartan diminish the clinical utility of B-type natriuretic peptide testing in acute cardiac care? 2016 12. Eur Heart J Acute Cardiovasc Care. 2017;6(4):321-328.
- 20 Zile MR, Claggett BL, Prescott MF, et al. Prognostic Implications of changes in N-Terminal Pro-B-Type Natriuretic Peptide in Patients With Heart Failure. J Am Coll Cardiol 2016;68:2425-2436.
- 21 Scirica BM, Braunwald E, Raz I, et al. Heart failure, saxagliptin, and diabetes mellitus: observations from the SAVOR-TIMI 53 randomized trial. Circulation. 2014;130:1579-88.
- 22 Bergmark BA, Morrow DA, Bhatt DL, et al. Natriuretic peptides versus a clinical history of heart failure for risk prediction in patients with diabetes. J Am Coll Cardiol. 2019; 73(19):681.
- 23 Bidadkosh A, Lambooy SPH, Heerspink HJ, et al. Predictive Properties of Biomarkers GDF-15, NTproBNP, and hs-TnT for Morbidity and Mortality in Patients With Type 2 Diabetes With Nephropathy. Diabetes Care. 2017;40(6):784-792.
- 24 Hamano K, Nakadaira I, Suzuki J, et al. N-terminal fragment of probrain natriuretic peptide is associated with diabetes microvascular complications in type 2 diabetes. Vasc Health Risk Manag. 2014;10:585-589.
- 25 Hillis GS, Welsh P, Chalmers J, et al. The relative and combined ability of high-sensitivity cardiac troponin T and N-terminal pro-B-type natriuretic peptide to predict cardiovascular events and death in patients with type 2 diabetes. Diabetes Care. 2014;37(1):295-303.

- 26 Bruno G, Landi A, Barutta F, et al. N-terminal probrain natriuretic peptide is a stronger predictor of cardiovascular mortality than Creactive protein and albumin excretion rate in elderly patients with type 2 diabetes: The Casale Monferrato population-based study. Diabetes Care. 2013;36(9):2677-2682.
- 27 Neuhold S, Resl M, Huelsmann M, et al. Repeat measurements of glycated haemoglobin A(1c) and N-terminal pro-B-type natriuretic peptide: divergent behaviour in diabetes mellitus. Eur J Clin Invest. 2011;41(12):1292-8.
- 28 Wolsk E, Claggett B, Diaz R, et al. Increases in Natriuretic Peptides Precede Heart Failure Hospitalization in Patients With a Recent Coronary Event and Type 2 Diabetes Mellitus. Circulation. 2017;136(16):1560-1562.
- 29 Jarolim P, White WB, Cannon CP, et al. Serial measurement of natriuretic peptides and cardiovascular outcomes in patients with type 2 diabetes in the EXAMINE trial. Diabetes Care. 2018; 41(7): 1510-1515.
- 30 Huelsmann M, Neuhold S, Resl M, et al. PONTIAC (NT-proBNP selected prevention of cardiac events in a population of diabetic patients without a history of cardiac disease): a prospective randomized controlled trial. J Am Coll Cardiol. 2013;62(15):1365-72.
- 31 Kemp Gudmundsdottir K, Fredriksson T, Svennberg E, et al. Stepwise mass screening for atrial fibrillation using N-terminal B-type natriuretic peptide: the STROKESTOP II study. Europace 2020;22(1):24-32.
- 32 De Bold AJ. Atrial Natriuretic Factor: A Hormone Produced by the Heart. Science 1985;230:767-770.
- 33 Valli N, Gobinet A, Bordenave L. Review of 10 years of the clinical use of brain natriuretic peptide in cardiology. J Lab Clin Med 1999:134:437-444.
- 34 Clerico A, Passino C, Franzini M, et al. Cardiac biomarker testing in the clinical labratory: Where do we stand? General overview of the methodology with special emphasis on natriuretic peptides. Clin Chimica Acta 2015;443:17-24.
- 35 De Bold AJ, Boerenstein HB, Veress AT, et al. A rapid and potent natriuretic response to intravenous injection of atrial extracts in rats. Life Sci 1981;28:89-94.
- 36 Epstein M, Loutzenhiser R, Friedland E, et al. Relationship of Increased Plasma Atrial Natriuretic Factor and Renal Sodium Handling During Immersion-induced Central Hypervolemia in Normal Humans. J Clin Invest 1987;79:738-745.
- 37 Eurlings Lw, Sanders-van Wijk S, van Kraaij DJ, et al. Risk stratification with the use of serial N-terminal pro-B-type natriuretic peptide measurements during admission and early after discharge in heart failure patients: post hoc analysis of the PRIMA study. J Card Fail 2014;20(12):881-890.
- 38 Salah K, Kok WE, Eurlings LW, et al. A novel discharge risk model for patients hospitalised for acute decompensated heart failure incorporating N-terminal pro-B-type natriuretic peptide levels: a European coLlaboration on Acute decompeNsated Heart Failure: ELAN-HF Score. Heart 2014;100(2):115-125.
- 39 Stienen S, Salah K, Dickhoff C, et al. N-Terminal Pro-B-Type Natriuretic Peptide (NT-proBNP) Measurements Until a 30% Reduction Is Attained During Acute Decompensated Heart Failure Admissions and Comparison With Discharge NT-proBNP Levels: Implications for In-Hospital Guidance of Treatment. J Card Fail 2015;21(11):930-934.
- 40 Stienen S, Salah K, Eurlings LW, et al. Challenging the two concepts in determining the appropriate pre-discharge N-terminal pro-brain natriuretic peptide treatment target in acute decompensated heart failure patients: absolute or relative discharge levels? Eur J Heart Fail 2015;17(9):936-944.
- 41 Savarese G, Musella F, D'Amore C, et al. Changes of natriuretic peptides predict hospital admissions in patients with chronic heart failure: a meta-analysis. JACC Heart Fail 2014;2(2):148-158.
- 42 Sargento L, Satendra M, Longo S, et al. Early NT-proBNP decrease with ivabradine in ambulatory patients with systolic heart failure. Clin Cardiol. 2013;36(11):677-682.



- 43 Masson S, Latini R, Anand IS, et al. Direct comparison of B-type natriuretic peptide (BNP) and amino-terminal proBNP in a large population of patients with chronic and symptomatic heart failure: the Valsartan Heart Failure (Val-HeFT) data. Clin Chem 2006;52:1528–1538.
- 44 Packer M, McMurray JJV, Desai Akshay S. et al. Angiotensin Receptor Neprilysin Inhibition Compared With Enalapril on the Risk of Clinical Progression in Surviving Patients With Heart Failure. Circulation 2015;6;131(1):54-61.
- 45 Solomon SD, Zile M, Pieske B. et al. The angiotensin receptor neprilysin inhibitor LCZ696 in heart failure with preserved ejection fraction: a phase 2 double-blind randomised controlled trial. Lancet 2012;380:1387-1395.
- 46 Huelsmann M, Neuhold S, Strunk G, et al. NT-proBNP has a high negative predictive value to rule-out short-term cardiovascular events in patients with diabetes mellitus. Eur Heart J 2008;29:2259-2264.
- 47 Prausmüller S, Resl M, Arfsten H, et al. Performance of the recommended ESC/EASD cardiovascular risk stratification model in comparison to SCORE and NT-proBNP as a single biomarker for risk prediction in type 2 diabetes mellitus. Cardiovasc Diabetol 2021;20(1):34.
- 48 Latini R, Masson S, Pirelli S, et al. Circulating cardiovascular biomarkers in recurrent atrial fibrillation: data from the GISSI-Atrial fibrillation trial. J Intern Med 2011;269(2):160-171.
- 49 Patton KK, Ellinor PT, Heckbert SR, et al. N-terminal pro-B-type natriuretic peptide is a major predictor of the development of atrial fibrillation: The Cardiovascular Health Study. Circulation 2009:120:1768-1774.
- 50 Seegers J, Zabel M, Gruter T, et al. Natriuretic peptides for the detection of paroxysmal atrial fibrillation. Open Heart 2015;2:e000182.
- 51 Hijazi Z, Lindbäck J, Alexander JH et al. The ABC (age, biomarkers, clinical history) stroke risk score: a biomarker-based risk score for predicting stroke in atrial fibrillation. Eur Heart J. 2016;37(20):1582-90.
- 52 Berg DD, Ruff CT, Jarolim P, et al. Performance of the ABC scores for Assessing the Risk of Stroke or Systemic Embolism and Bleeding in Patients with Atrial Fibrillation in ENGAGE AF-TIMI 48. Circulation. 2019;139(6):760-771.
- 53 Zamorano JL, Lancellotti P, Rodriguez Muñoz D et al. 2016 ESC Position Paper on cancer treatments and cardiovascular toxicity developed under the auspices of the ESC Committee for Practice Guidelines: The Task Force for cancer treatments and cardiovascular toxicity of the European Society of Cardiology (ESC). Eur Heart J. 2016;37(36):2768-2801.
- 54 Brune K, Katus HA, Moecks J, et al. N-terminal pro-B-type natriuretic peptide concentrations predict the risk of cardiovascular adverse events from antiinflammatory drugs: apilot trial. Clin Chem 2008;54(7):1149-1157.
- 55 Bojunga J, Sarrazin C, Hess G, et al. Elevated plasma levels of Nterminal pro-brain natriuretic peptide in patients with chronic hepatitis C during interferon-based antiviral therapy. World J Gastroenterol 2006;12(36):5875-5877.
- 56 Slordal L, Spigset O. Heart failure induced by non-cardiac drugs. Drug safety 2006;29(7):567-586.
- 57 Giannitsis E. Rationale for testing the cardiovascular risk for patients with COX-2 inhibitors on the basis of biomarker NT-proBNP. Clin Lab 2005:51(1-2):63-83.
- 58 Häupl T, Burmester GR, Giannitsis E, et al. N-terminal prohormone brain natriuretic peptide: a biomarker for detecting cardiovascular risks in patients with rheumatoid arthritis or osteoarthritis? Ann Rheum Dis 2007;66(6):838-839.
- 59 Duceppe E, Parlow J, MacDonald P, et al. Canadian cardiovascular society guidelines on perioperative cardiac risk assessment and management for patients undergoing noncardiac surgery. Can J Cardiol. 2017;33(1):17-32.
- 60 Natriuretic Peptides Studies Collaboration. Natriuretic peptides and integrated risk assessment for cardiovascular disease: an individualparticipant-data meta-analysis. Lancet Diabetes Endocrinol. 2016;4(10):840-849.

- 61 Cowie MR, Jourdain P, Maisel A, et al. Clinical applications of B-type natriuretic peptide (BNP) testing. Eur Heart J 2003;24,1710-1718.
- 62 Nielsen LS, Svanegaard J, Klitgaard NA, et al. N-terminal pro-brain natriuretic peptide for discriminating between cardiac and non-cardiac dyspnea. Eur J Heart Fail 2004;6:63-70.
- 63 Tzikas S, Keller T, Wild PS, et al. Midregional pro-atrial natriuretic peptide in the general population/insighs from the Gutenberg Health Study. Clin Chem Lab Med. 2013;51(5):1125-33.
- 64 Liao JN, Chao TF, Kuo JY, et al. Age, sex and blood pressure-related influences on reference values of left atrial deformation and mechanics from a large-scale Asian population. Circ Cardiovasc Imaging. 2017;10(10):e006077.
- 65 Albers S, Mir TS, Haddad M, et al. N-Terminal pro-brain natriuretic peptide: normal ranges in the pediatric population including method comparison and interlaboratory variability. Clin Chem Lab Med 2006;44(1):80-85.
- 66 Al-Barjas M, Nair D, Morris R, et al. How can the role of N terminal pro B Natriuretic Peptide (NT-proBNP) be optimised in heart failure screening? A prospective observational comparative study. Eur J Heart Fail 2004;3:51 Supplement 1.
- 67 Gustafsson F, Badskjær J, Hansen F, et al. Value of N-Terminal proBNP in the Diagnosis of Left Ventricular Systolic Dysfunction in Primary Care Patients Referred for Echocardiography. Heart Drug 2003.3:141-146.
- 68 Zaphiriou A, et al. The diagnostic accuracy of plasma BNP and NTproBNP in patients referred from primary care with suspected heart failure: results of the UK natriuretic peptide study.
- 69 Gustafsson F, et al. Diagnostic and prognostic performance of Nterminal ProBNP in primary care patients with suspected heart failure. J Card Fail. 2005 Jun;11(5 Suppl):S15-20.
- 70 Taylor CJ, et al. Primary care REFerral for EchocaRdiogram (REFER) in heart failure: a diagnostic accuracy study. Br J Gen Pract. 2017 Feb;67(655):e94-e102.
- 71 Ibrahim I, Kuan WS, Frampton C, et al. Superior performance of Nterminal pro brain natriuretic peptide for diagnosis of acute decompensated heart failure in an Asian compared with a Western setting. Eur J Heart Fail. 2017 feb;19(2):209-217.
- 72 Bablok W, Passing H, Bender R, et al. A general regression procedure for method transformation. Application of linear regression procedures for method comparison studies in clinical chemistry, Part III. J Clin Chem Clin Biochem 1988 Nov;26(11):783-790.

For further information, please refer to the appropriate operator's manual for the analyzer concerned, the respective application sheets and the Method Sheets of all necessary components (if available in your country).

A point (period/stop) is always used in this Method Sheet as the decimal separator to mark the border between the integral and the fractional parts of a decimal numeral. Separators for thousands are not used.

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

The Summary of Safety & Performance Report can be found here: https://ec.europa.eu/tools/eudamed

Symbols

Roche Diagnostics uses the following symbols and signs in addition to those listed in the ISO 15223-1 standard (for USA: see dialog.roche.com for definition of symbols used):

CONTENT Contents of kit

SYSTEM Analyzers/Instruments on which reagents can be used

REAGENT Reagent

CALIBRATOR Calibrator

Volume for reconstitution

Global Trade Item Number

GTIN



COBAS, COBAS E, ELECSYS and PRECICONTROL are trademarks of Roche. INTRALIPID is a trademark of Fresenius Kabi AB.

All other product names and trademarks are the property of their respective owners. Additions, deletions or changes are indicated by a change bar in the margin.

© 2023, Roche Diagnostics





Roche Diagnostics GmbH, Sandhofer Strasse 116, D-68305 Mannheim www.roche.com

+800 5505 6606

