

## Triglycerides

### Order information

REF		CONTENT		Analyzer(s) on which <b>cobas c</b> pack(s) can be used
05171407190*	05171407500	Triglycerides (800 tests)	System-ID 05 6710 7	<b>cobas c</b> 701/702
05171407214*	05171407500	Triglycerides (800 tests)	System-ID 05 6710 7	<b>cobas c</b> 701/702

Materials required (but not provided):

10759350190	Calibrator f.a.s. (12 x 3 mL)	Code 401	
12149435122	Precinorm U plus (10 x 3 mL)	Code 300	
12149443122	Precipath U plus (10 x 3 mL)	Code 301	
05117003190	PreciControl ClinChem Multi 1 (20 x 5 mL)	Code 391	
05947626190	PreciControl ClinChem Multi 1 (4 x 5 mL)	Code 391	
05117216190	PreciControl ClinChem Multi 2 (20 x 5 mL)	Code 392	
05947774190	PreciControl ClinChem Multi 2 (4 x 5 mL)	Code 392	
05172152190	Diluent NaCl 9 % (119 mL)	System-ID 08 6869 3	

\* Some kits shown may not be available in all countries.

### English

#### System information

TRIGL: ACN 8781

#### Intended use

In vitro test for the quantitative determination of triglycerides in human serum and plasma on **cobas c** systems.

#### Summary

Triglyceride measurements, performed with this assay in human serum and plasma are used as an aid in identifying patients at risk of developing atherosclerosis and for the diagnosis of dyslipidemias.

Triglycerides are esters of the trihydric alcohol glycerol with 3 long-chain fatty acids.<sup>1</sup> They are partly synthesized in the liver and partly ingested in food. Triglycerides are water-insoluble molecules and are carried in the circulation in water-soluble complexes called lipoproteins. The plasma triglyceride level reflects the concentration of the triglyceride-carrying lipoproteins VLDL (very-low-density lipoproteins) and chylomicrons.<sup>2</sup> Chylomicrons are primarily involved in the absorption and delivery of dietary fat while VLDLs deliver endogenous lipids to other tissues.

Triglycerides are considered a risk factor for atherosclerotic cardiovascular disease.<sup>3</sup> Cardiovascular risk is increased when fasting triglycerides are > 1.7 mmol/L (> 150 mg/dL). Individuals with triglycerides > 2.3 mmol/L (> 200 mg/dL) are considered at high risk. The determination of triglycerides is utilized in the diagnosis of patients having diabetes mellitus, nephrosis, liver obstruction, lipid metabolism disorders and numerous other endocrine diseases.<sup>4</sup>


Elevated levels of plasma triglycerides are also associated with an increased risk of acute pancreatitis and aortic valve stenosis.<sup>5</sup>

The enzymatic triglycerides assay as described by Eggstein and Kreutz still required saponification with potassium hydroxide. Numerous attempts were subsequently made to replace alkaline saponification by enzymatic hydrolysis with lipase.<sup>6</sup> Bucolo and David tested a lipase/protease mixture; Wahlefeld used an esterase from the liver in combination with a particularly effective lipase from *Rhizopus arrhizus* for hydrolysis.<sup>7, 8</sup>


This method is based on the work by Wahlefeld using a lipoprotein lipase from microorganisms for the rapid and complete hydrolysis of triglycerides to glycerol followed by oxidation to dihydroxyacetone phosphate and hydrogen peroxide. The hydrogen peroxide produced then reacts with 4-aminophenazone and 4-chlorophenol under the catalytic action of peroxidase to form a red dyestuff (Trinder endpoint reaction). The color intensity of the red dyestuff formed is directly proportional to the triglyceride concentration and can be measured photometrically.<sup>9, 10</sup>

#### Test principle<sup>10</sup>


Enzymatic colorimetric test.

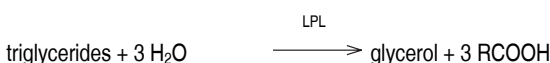

GK

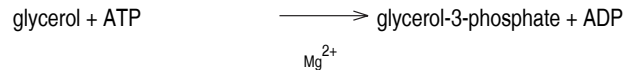

GPO

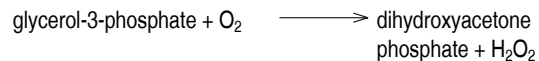

peroxidase



#### Reagents - working solutions

**R1** PIPES buffer: 50 mmol/L, pH 6.8; Mg<sup>2+</sup>: 40 mmol/L; sodium cholate: 0.20 mmol/L; ATP: ≥ 1.4 mmol/L; 4-aminophenazone: ≥ 0.13 mmol/L; 4-chlorophenol: 4.7 mmol/L; lipoprotein lipase (*Pseudomonas spec.*): ≥ 83 μkat/L; glycerol kinase (*Bacillus stearothermophilus*): ≥ 3 μkat/L; glycerol phosphate oxidase (*E. coli*): ≥ 41 μkat/L; peroxidase (horseradish): ≥ 1.6 μkat/L; preservative, stabilizers

R1 is in position B and C.

#### Precautions and warnings

For in vitro diagnostic use for health care professionals. Exercise the normal precautions required for handling all laboratory reagents.

Infectious or microbial waste:

Warning: handle waste as potentially biohazardous material. Dispose of waste according to accepted laboratory instructions and procedures.

Environmental hazards:

Apply all relevant local disposal regulations to determine the safe disposal.

Safety data sheet available for professional user on request.

#### Reagent handling

Ready for use

#### Storage and stability

Shelf life at 2-8 °C:

See expiration date on **cobas c** pack label.

On-board in use and refrigerated on the analyzer:

2 weeks

On-board on the Reagent Manager:

24 hours

**Specimen collection and preparation**

For specimen collection and preparation only use suitable tubes or collection containers.

Only the specimens listed below were tested and found acceptable.

Serum

Plasma: Li-heparin, K<sub>2</sub>-EDTA plasma.

The sample types listed were tested with a selection of sample collection tubes that were commercially available at the time of testing, i.e. not all available tubes of all manufacturers were tested. Sample collection systems from various manufacturers may contain differing materials which could affect the test results in some cases. When processing samples in primary tubes (sample collection systems), follow the instructions of the tube manufacturer.

Centrifuge samples containing precipitates before performing the assay.

See the limitations and interferences section for details about possible sample interferences.

Stability in serum:	2 days at 20-25 °C <sup>11</sup>
	10 days at 2-8 °C <sup>12</sup>
	3 months at -20 °C (± 5 °C) <sup>13</sup>
	several years at -70 °C (± 5 °C) <sup>13</sup>

Freeze only once.

Stability in plasma:	2 days at 20-25 °C <sup>11</sup>
	15 days at 2-8 °C <sup>14</sup>
	3 months at -20 °C (± 5 °C) <sup>13</sup>
	several years at -70 °C (± 5 °C) <sup>13</sup>

Freeze only once.

**Materials provided**

See "Reagents – working solutions" section for reagents.

**Materials required (but not provided)**

See "Order information" section

General laboratory equipment

**Assay**

For optimum performance of the assay follow the directions given in this document for the analyzer concerned. Refer to the appropriate operator's manual for analyzer-specific assay instructions.

The performance of applications not validated by Roche is not warranted and must be defined by the user.

**Application for serum and plasma****cobas c 701/702 test definition**

Assay type	1-Point		
Reaction time / Assay points	10 / 38		
Wavelength (sub/main)	700 / 505 nm		
Reaction direction	Increase		
Units	mmol/L (mg/dL, g/L)		
Reagent pipetting		Diluent (H <sub>2</sub> O)	
R1	120 µL	28 µL	
<b>Sample volumes</b>	<b>Sample</b>	<b>Sample dilution</b>	
		<b>Sample</b>	<b>Diluent (NaCl)</b>
Normal	2 µL	–	–
Decreased	4 µL	15 µL	135 µL
Increased	4 µL	–	–

**Calibration**

Calibrators	S1: H <sub>2</sub> O
	S2: C.f.a.s.

Calibration mode	Linear
Calibration frequency	2-point calibration
	- after reagent lot change
	- as required following quality control procedures
	Blank calibration
	- after cassette change

Calibration interval may be extended based on acceptable verification of calibration by the laboratory.

Traceability: This method has been standardized against the ID/MS method.

**Quality control**

For quality control, use control materials as listed in the "Order information" section.

In addition, other suitable control material can be used.

The control intervals and limits should be adapted to each laboratory's individual requirements. Values obtained should fall within the defined limits. Each laboratory should establish corrective measures to be taken if values fall outside the defined limits.

Follow the applicable government regulations and local guidelines for quality control.

**Calculation**

**cobas c** systems automatically calculate the analyte concentration of each sample.

Conversion factors:

$$\text{mmol/L} \times 88.5 = \text{mg/dL} \qquad \text{mmol/L} \times 0.885 = \text{g/L}$$

$$\text{mg/dL} \times 0.0113 = \text{mmol/L} \qquad \text{mg/dL} \times 0.01 = \text{g/L}$$

**Limitations - interference**

Criterion: Recovery within ± 10 % of initial values at triglyceride levels of 2.3 mmol/L (203 mg/dL).

Icterus:<sup>15</sup> No significant interference up to an I index of 10 for conjugated bilirubin and 10 for unconjugated bilirubin (approximate conjugated bilirubin concentration: 171 µmol/L or 10 mg/dL; approximate unconjugated bilirubin concentration: 171 µmol/L or 10 mg/dL).

Hemolysis:<sup>15</sup> No significant interference up to an H index of 700 (approximate hemoglobin concentration: 434 µmol/L or 700 mg/dL).

Lipemia:<sup>15</sup> The L index correlates with sample turbidity but not with triglycerides level. Extremely lipemic samples (triglycerides greater than 3000 mg/dL) can produce normal results.<sup>16</sup>

Prozone Check: The flag > Kin is an indicator for extremely high triglyceride concentrations in the sample. False low results are due to oxygen depletion during assay reaction.

Endogenous unesterified glycerol in the sample will falsely elevate serum triglycerides.

Drugs: No interference was found at therapeutic concentrations using common drug panels.<sup>17,18</sup>

Exception: Ascorbic acid and calcium dobesilate cause artificially low triglyceride results. Intralipid is directly measured as analyte in this assay and leads to high triglyceride results.

Dicynone (Etamsylate) at therapeutic concentrations may lead to false-low results.<sup>19</sup>

Acetaminophen intoxications are frequently treated with N-Acetylcysteine. N-Acetylcysteine at a plasma concentration above 166 mg/L and the Acetaminophen metabolite N-acetyl-p-benzoquinone imine (NAPQI) independently may cause falsely low results.

Venipuncture should be performed prior to the administration of Metamizole. Venipuncture immediately after or during the administration of Metamizole may lead to falsely low results. A significant interference may occur at plasma Metamizole concentrations above 0.05 mg/mL.

In very rare cases, gammopathy, in particular type IgM (Waldenström's macroglobulinemia), may cause unreliable results.<sup>20</sup>

## Triglycerides

For diagnostic purposes, the results should always be assessed in conjunction with the patient's medical history, clinical examination and other findings.

### ACTION REQUIRED

**Special Wash Programming:** The use of special wash steps is mandatory when certain test combinations are run together on **cobas c** systems. All special wash programming necessary for avoiding carry-over is available via the **cobas** link, manual input is required in certain cases. The latest version of the carry-over evasion list can be found with the NaOHD/SMS/SmpCln1+2/SCCS Method Sheet and for further instructions refer to the operator's manual.

**Where required, special wash/carry-over evasion programming must be implemented prior to reporting results with this test.**

### Limits and ranges

#### Measuring range

0.1-10.0 mmol/L (8.85-885 mg/dL)

Determine samples having higher concentrations via the rerun function. Dilution of samples via the rerun function is a 1:5 dilution. Results from samples diluted using the rerun function are automatically multiplied by a factor of 5.

#### Lower limits of measurement

##### Lower detection limit of the test

0.1 mmol/L (8.85 mg/dL)

The lower detection limit represents the lowest measurable analyte level that can be distinguished from zero. It is calculated as the value lying 3 standard deviations above that of the lowest standard (standard 1 + 3 SD, repeatability, n = 21).

Values below the lower detection limit (< 0.1 mmol/L) will not be flagged by the instrument.

#### Expected values according to NCEP<sup>21</sup>

Normal range: < 1.70 mmol/L (< 150 mg/dL).

**Clinical interpretation** according to the recommendations of the European Atherosclerosis Society:<sup>22</sup>

	mmol/L	mg/dL	Lipid metabolism disorder
Cholesterol	< 5.18	< 200	No
Triglycerides	< 2.26	< 200	No
Cholesterol	5.18-7.77	200-300	Yes if HDL-cholesterol < 0.9 mmol/L (< 35 mg/dL)
Cholesterol	> 7.77	> 300	Yes
Triglycerides	> 2.26	> 200	Yes

**Note:** If the free glycerol is to be taken into account, the 0.11 mmol/L (10 mg/dL) must be subtracted from the triglycerides value obtained.<sup>13</sup>

Each laboratory should investigate the transferability of the expected values to its own patient population and if necessary determine its own reference ranges.

### Specific performance data

Representative performance data on the analyzers are given below. Results obtained in individual laboratories may differ.

#### Precision

Precision was determined using human samples and controls in an internal protocol with repeatability (n = 21) and intermediate precision (3 aliquots per run, 1 run per day, 21 days). The following results were obtained:

Repeatability	Mean	SD	CV
	mmol/L (mg/dL)	mmol/L (mg/dL)	%
Precinorm U	1.32 (117)	0.01 (1)	0.9
Precipath U	2.40 (212)	0.01 (1)	0.6
Human serum A	4.57 (404)	0.03 (3)	0.7
Human serum B	6.39 (566)	0.06 (5)	0.9
Human serum C	9.21 (815)	0.05 (4)	0.6

Intermediate precision	Mean	SD	CV
	mmol/L (mg/dL)	mmol/L (mg/dL)	%
Precinorm U	1.39 (123)	0.03 (3)	2.0
Precipath U	2.33 (206)	0.04 (4)	1.6
Human serum 3	1.18 (104)	0.02 (2)	1.9
Human serum 4	2.95 (261)	0.05 (4)	1.8

Results for intermediate precision were obtained on the **cobas c 501** analyzer.

#### Method comparison

Triglycerides values for human serum and plasma samples obtained on a **cobas c 701** analyzer (y) were compared with those determined using the corresponding reagent on a **cobas c 501** analyzer (x).

Sample size (n) = 81

Passing/Bablok <sup>23</sup>	Linear regression
y = 1.000x - 0.010 mmol/L	y = 1.001x - 0.010 mmol/L
t = 0.989	r = 1.000

The sample concentrations were between 0.47 and 9.53 mmol/L (41.6 and 843 mg/dL)

#### References

- Gordon MH. FATS I Classification. In: Benjamin Caballero, editor. Encyclopedia of Food Sciences and Nutrition, 2nd edition, Academic Press 2003, p. 2287-2292.
- Berglund L, Brunzell JD, Goldberg AC, et al. Evaluation and treatment of hypertriglyceridemia: an Endocrine Society clinical practice guideline. J Clin Endocrinol Metab 2012;97(9):2969-2989.
- Mach F, Baigent C, Catapano AL, et al. 2019 ESC/EAS Guidelines for the management of dyslipidaemias: lipid modification to reduce cardiovascular risk. Eur Heart J 2020;41(1):111-88.
- Rader DJ, Kathiresan S. Disorders of Lipoprotein Metabolism. In: Jameson JL, Fauci AS, Kasper DL, Hauser SL, Longo DL, Loscalzo J, editors. Harrison's Principles of Internal Medicine, 20e. New York, NY: McGraw-Hill Education; 2018.
- Meeusen JW, Ueda M, Nordestgaard BG, et al. Lipids and lipoproteins. In: Rifai N, Chiu RWK, Young I, Burnham CAD, Wittwer CT, editors. Tietz Textbook of Laboratory Medicine, Saunders Elsevier, Philadelphia, 7th edition, 2023, chapter 36, p. 354-414.e10.
- Eggstein M, Kreutz FH. A new determination of the neutral fats in blood serum and tissue. I. Principles, procedure, and discussion of the method. Klin Wschr 1966;44(5):262-267.
- Bucolo G, David H. Quantitative determination of serum triglycerides by the use of enzymes. Clin Chem 1973;19(5):476-482.
- Wahlefeld AW, Bergmeyer HU, eds. Methods of Enzymatic Analysis. 2nd English ed. New York, NY:Academic Press Inc 1974;1831.
- Trinder P. Determination of glucose in blood using glucose oxidase with an alternative oxygen acceptor. Ann Clin Biochem 1969;6:24-27.
- Siedel J, Schmuck R, Staepels J, et al. Long term stable, liquid ready-to-use monoreagent for the enzymatic assay of serum or plasma triglycerides (GPO-PAP method). AACC Meeting Abstract 34. Clin Chem 1993;39:1127.
- WHO Publication: Use of anticoagulants in diagnostic laboratory investigations, WHO/DIL/LAB/99.1 Rev.2:Jan 2002.
- Evans K, Mitcheson J, Laker M. Effect of Storage at 4 °C and -20 °C on Lipid, Lipoprotein, and Apolipoprotein Concentrations. Clin Chem. 1995;41:392-396.
- Tietz NW, ed. Clinical Guide to Laboratory Tests, 3rd ed. Philadelphia, PA: WB Saunders Company 1995;610-611.
- Kronenberg F, Lobentanz EM, König P, et al. Effect of sample storage on the measurement of lipoprotein[a], apolipoproteins B and A-IV, total and high density lipoprotein cholesterol and triglycerides. J Lipid Res. 1994 Jul;35(7):1318-28.

# TRIGL

## Triglycerides

- 15 Glick MR, Ryder KW, Jackson SA. Graphical Comparisons of Interferences in Clinical Chemistry Instrumentation. Clin Chem 1986;32:470-475.
- 16 Shephard MDS, Whiting MJ. Falsely low estimation of triglycerides in lipemic plasma by the enzymatic triglyceride method with modified Trinder's chromogen. Clin Chem 1990;36(2):325-329.
- 17 Breuer J. Report on the Symposium "Drug effects in Clinical Chemistry Methods". Eur J Clin Chem Clin Biochem 1996;34:385-386.
- 18 Sonntag O, Scholer A. Drug interference in clinical chemistry: recommendation of drugs and their concentrations to be used in drug interference studies. Ann Clin Biochem 2001;38:376-385.
- 19 Dastych M, Wiewiorka O, Benovska M. Ethamsylate (Dicynone) Interference in Determination of Serum Creatinine, Uric Acid, Triglycerides, and Cholesterol in Assays Involving the Trinder Reaction; In Vivo and In Vitro. Clin Lab 2014;60:1373-1376.
- 20 Bakker AJ, Mücke M. Gammopathy interference in clinical chemistry assays: mechanisms, detection and prevention. Clin Chem Lab Med 2007;45(9):1240-1243.
- 21 U.S. DEPARTMENT OF HEALTH AND HUMAN SERVICES Public Health Service, NIH Publication No. 01-3305, May 2001.
- 22 Study Group, European Atherosclerosis Society. Strategies for the prevention of coronary heart disease: A policy statement of the European Atherosclerosis Society. European Heart Journal 1987;8:77-88.
- 23 Bablok W, Passing H, Bender R, et al. A general regression procedure for method transformation. Application of linear regression procedures for method comparison studies in clinical chemistry, Part III. J Clin Chem Clin Biochem 1988 Nov;26(11):783-790.

A point (period/stop) is always used in this Method Sheet as the decimal separator to mark the border between the integral and the fractional parts of a decimal numeral. Separators for thousands are not used.

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

### Symbols

Roche Diagnostics uses the following symbols and signs in addition to those listed in the ISO 15223-1 standard (for USA: see [navifyportal.roche.com](http://navifyportal.roche.com) for definition of symbols used):

CONTENT	Contents of kit
→	Volume for reconstitution
GTIN	Global Trade Item Number
Rx only	For USA: Caution: Federal law restricts this device to sale by or on the order of a physician.

COBAS, NAVIFY, PRECICONTROL, PRECINORM and PRECIPATH are trademarks of Roche.

All other product names and trademarks are the property of their respective owners.

Additions, deletions or changes are indicated by a change bar in the margin.

© 2024, Roche Diagnostics



Roche Diagnostics GmbH, Sandhofer Strasse 116, D-68305 Mannheim  
[www.roche.com](http://www.roche.com)  
 +800 5505 6606

